## PEAK FLOW METER

**en CARE AND USE OF YOUR PEAK FLOW METER**  
It is important to keep your Peak Flow Meter clean when in use. It should be cleaned at least once every six months and thoroughly if it has not been used for some time. The mouthpiece is detachable and should be disinfected before each use using a proprietary disinfectant solution. The Mini-Wright Peak Flow Meter is designed for single patient use. For healthcare professionals, cleaning instructions for multi patient use can be obtained from Clement Clarke International Limited, see address overleaf.  
**PERFORMANCE ACCURACY.** The Mini-Wright standard range Peak Flow Meter meets NAEP guidelines and UK Department of Health specification no. 51.  
If you have received a **Black Scale** meter: variations <±10% with Standard Wright Peak Flow Meter.  
If you have received a **Purple Scale** meter, stating "ATS 94": Variations <±5% or 10 L/min (whichever is the greater) on ATS waveform generator using PW24 multiples.  
△ Fungal growths may occur if the instrument has not been cleaned for some time.

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**en YOUR PEAK FLOW METER**  
Your Mini-Wright Peak Flow Meter will help you to monitor your asthma by measuring your peak flow. This leaflet explains how to use your Peak Flow Meter, and how to record the readings. With careful use the Mini-Wright would be expected to last for three years.

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