

DISCOVER THE **OXY** EFFECT - NOW ALSO IN **XL**

## Incidin™ OxyFoam S & OxyWipe S

Sporicidal ready-to-use cleaner and disinfectant with Hi-speed H<sub>2</sub>O<sub>2</sub>™

The advanced formulation of Incidin™ OxyFoam S & Incidin™ OxyWipe S provides the benefits of a patented\* hydrogen peroxide solution with broad spectrum efficacy.

They are effective against a wide range of pathogens and leave no active residues due to their rapid evaporation rate.

Incidin™ OxyFoam S & OxyWipe S provide:

- ▲ Hi-speed H<sub>2</sub>O<sub>2</sub>™
- ▲ Active ingredient decomposing into water and oxygen
- ▲ Sporicidal & virucidal offering for all surfaces
- ▲ Excellent material compatibility



# Incidin™ OxyFoam S & OxyWipe S

Sporicidal ready-to-use cleaner and disinfectant with Hi-speed H<sub>2</sub>O<sub>2</sub>™

**3 MIN<sup>1</sup>** Sporicidal against **C. difficile** (SUSPENSIONS TEST)

**2 MIN<sup>1</sup>** Virucidal (CLEAN CONDITIONS)

**1 MIN<sup>1</sup>** Bactericidal (EN 13727 / 16615)


Features	Benefits
<b>EFFECTIVE</b>	Broad spectrum efficacy Short contact times
<b>SAFE</b>	Active ingredient decomposes into water and oxygen. Excellent material compatibility
<b>CONVENIENT</b>	Cleans & disinfects in one step Ready to use formats

## MICROBIOLOGICAL AND VIROLOGICAL TEST RESULTS


EFFICACY UNDILUTED	OxyFoam S CONTACT TIME	OxyWipe S CONTACT TIME
Disinfection of surfaces acc. to VAH method 2015 (incl. EN 16615), dirty conditions		
Bactericidal, yeasticidal	5 min	15 min
Fungicidal	5 min	30 min
Mycobactericidal	15 min	30 min
Tested acc. RKI recommendation 01/2004*		
Effective against enveloped viruses acc. to RKI recommendation 01/2004*		
Noro (MNV)*	30 min	60 min
Adeno*	30 sec	1 min
Polyoma SV40*	30 sec	30 sec
Euro-Norms (clean & dirty conditions)		
EN 13727 (bactericidal)	1 min	2 min
EN 13624 (yeasticidal)	1 min	15 min
EN 13624 (fungicidal)	5 min	30 min
EN 14348 (tuberculocidal, mycobactericidal)	5 min	30 min
EN 14476 (virucidal)	2 min** / 30 min	30 min**
EN 14476 Adeno	30 sec	30 sec
EN 14476 (limited spectrum virucidal)	1 min (clean conditions) 15 min (dirty conditions)	15 min
EN 14476 Noro	1 min (clean conditions) 15 min (dirty conditions)	15 min
EN 13704 (sporicidal)**	30 min	60 min
EN 13704 (C. difficile)**	3 min	15 min
Euro-Norms (dirty conditions) - 4-field-test		
EN 16615 (bactericidal)	1 min	2 min
EN 16615 (yeasticidal)	1 min	1 min
EN 16615 (fungicidal, tuberculocidal, mycobactericidal)	5 min	5 min
Sporicidal (method EN 16615)**	15 min	30 min
C. difficile (method EN 16615)**	5 min	5 min

\* DVV 2015 (suspension test) \*\* clean conditions

## INCIDIN™ OXYFOAM S INSTRUCTIONS FOR USE

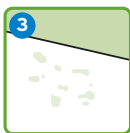

- 1 Wet surface
- 2 Wipe the surface ensuring it is completely wet
- 3 Let dry

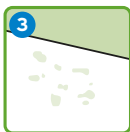

- 1 Apply to clean wipe
- 2 Wipe the surface ensuring it is completely wet
- 3 Let dry

### EUROPEAN HEADQUARTERS: ECOLAB EUROPE GMBH

Richtstrasse 7  
8304 Wallisellen  
Switzerland  
+41(0) 44-877-2001  
www.ecolab.com



**ECOLAB DEUTSCHLAND GMBH**  
Ecolab-Allee 1  
40789 Monheim am Rhein  
+49 (0) 2173-599-1900



EFFECTIVE.  
SAFE.  
CONVENIENT.

### APPLICATION AREA

Fast acting, sporicidal ready-to-use surface cleaning and disinfection



▲ acc. Directive 93/42/EEC: for medical devices.

▲ Biocide: for general surfaces.



### INSTRUCTIONS FOR USE

**Incidin™ OxyFoam S:** Wet surface, ensuring the surface is completely wet. Wipe the surface with a clean wipe and leave to dry. Alternatively, apply the product to a clean wipe, wipe the surface and leave to dry. Do not rinse.

**Incidin™ OxyWipe S:** Take single wipe. Wet surface thoroughly, ensuring contact with the entire surface. Let it dry and do not rinse. Close lid after use.

The use of a cleaning and disinfection product does not replace regular cleaning. For reprocessing of medical devices always observe manufacturers' instructions for use, incl. material compatibility. Terminal disinfection of semi-critical medical devices (i.e. probes & TEE probes): If immersion is not possible, disinfect the device by wiping as described above. Rinse thoroughly with water of drinking quality after use in this case. Do not use on surfaces sensitive to oxidative agents such as marble, copper and brass.

### INGREDIENTS

100 g contains the following active:

**Incidin™ OxyFoam S:** 1.5 g Hydrogen peroxide

**Incidin™ OxyWipe S:** 1.5 g Hydrogen peroxide

### PACKAGING AND ORDERING INFORMATION

PRODUCT	OUTER SIZE	ORDER CODE
Incidin™ OxyFoam S	6 x 750ml	
Incidin™ OxyFoam S	2 x 5L	
Incidin™ OxyWipe S	6 x 100 (20 x 20 cm)	
Incidin™ OxyWipe S XL	6 x 50 (25 x 37 cm)	
drain cock (DIN 51) for 5L canister	1 pcs	10033873
wall mounted dispenser for Incidin™ OxyWipe S	6 x 1 pcs	10033877

### STORAGE

Product should be stored in the original container below 25 °C in a cool, dry location out of direct sunlight.

Shelf life after first opening:

**Incidin™ OxyFoam S:** End of expiry date

**Incidin™ OxyWipe S:** 3 months

### References

1. Incidin™ OxyFoam S

USE BIOCIDES SAFELY. ALWAYS READ THE LABEL AND PRODUCT INFORMATION BEFORE USE. FOR PROFESSIONAL USE ONLY.