

seca CT 320/321 and seca cardio 150

Operating Manual

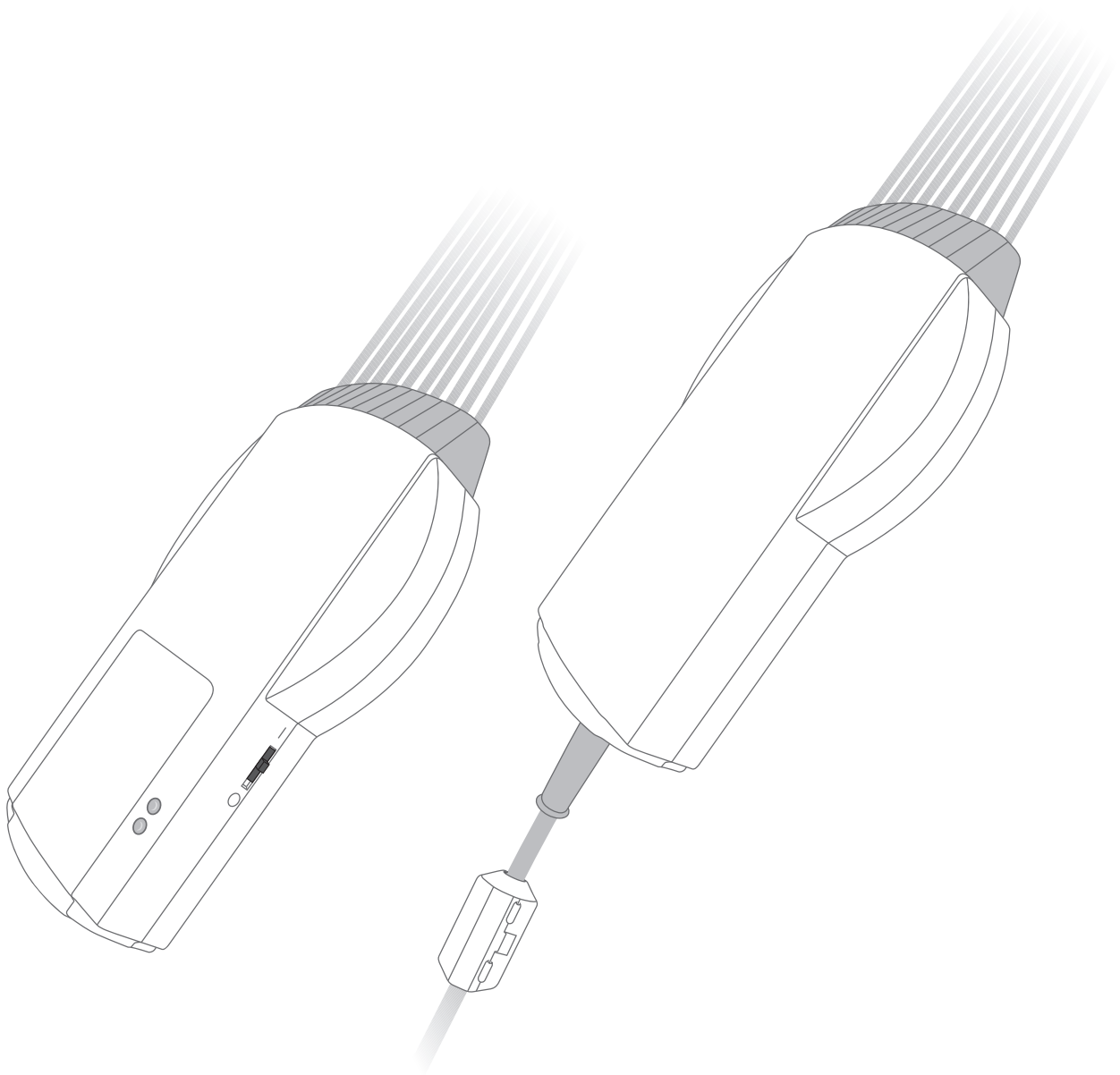






Table of contents

01	Introduction	
	01.1 Symbols on the device	4
	01.2 Intended use	4
	01.3 Symbols used in this manual	5
02	Safety instructions	
	02.1 General notes	6
	02.2 Operator instructions and user requirements	6
	02.3 Safety installations and safe working	7
	02.4 Information on EMC (Electromagnetic Compatibility)	11
	02.5 Maintenance (regular safety checks)	12
	02.6 Disclaimer	12
	02.7 Warranty	13
03	Description of device, initial operation	
	03.1 Part names, components for ECG recording	14
	03.2 Operation of seca CT 321 (with Bluetooth connectivity)	16
	03.3 seca cardio 150 – basic program structure	17
	03.4 Connecting the ECG device to the PC	
	03.4.1 ECG device with USB port	18
	03.4.2 ECG device with Bluetooth connection	19
	03.4.3 Selecting ECG device for resting ECG	21
04	Hygiene	
	04.1 Cleaning and disinfection	22
	04.2 Recommended cleaning agents and disinfectants	23

05	Carrying out an examination	
05.1	Preparing the patient, examination procedure.....	24
05.2	Electrode application.....	25
05.3	First work steps in seca cardio 150.....	26
05.4	Recording resting ECG	
05.4.1	Monitoring and electrode control.....	28
05.4.2	Starting and ending a resting ECG.....	28
05.4.3	Editing options during the recording process.....	29
05.4.4	Online ECG printing.....	29
06	Working with the evaluation	
06.1	Opening the evaluation.....	30
06.2	Resting ECG evaluation	
06.2.1	Screen pages within an evaluation.....	32
06.2.2	Comparing several evaluations from one patient.....	33
06.2.3	Note on navigation in the evaluation.....	33
06.2.4	Overview of evaluation pages.....	34
06.3	Writing the report.....	42
06.4	Ending the evaluation.....	43
07	Product Information	
07.1	Technical data and system requirements.....	44
07.2	Manufacturer declaration on EMC (Electromagnetic Compatibility) according to IEC 60601-1-2:2007.....	46
07.3	EC Declaration of Conformity.....	49
07.4	Shutdown, storage and transport.....	51
07.5	Disposal.....	51
07.6	Keyboard navigation and shortcuts.....	52

01 Introduction

01.1 Symbols on the device

	Device conforms with the standards and guidelines of the EU
	Observe the Operating Manual
	Separate collection of electrical and electronic equipment, do not dispose with domestic waste
	Protection class classification of medical electrical equipment according to IEC 60601-1 (type BF, defibrillation protected)

01.2 Intended use

seca CT 320/321 is a 12-channel PC ECG device which is designed for recording, analysing and evaluating ECG recordings in medical practices and hospitals.

seca CT 320/321 is perfectly safe for patients with a pacemaker. The ECG recording is not affected by pacemaker pulses. seca CT 320/321 is not suitable for intracardiac use.

01.3 Symbols used in this manual

This Operating Manual uses the following symbols to indicate important information, comments and tips:

ACTIONS THAT ARE PROHIBITED
or not allowed under any circumstances!



WARNING
used to indicate situations which, if not avoided, may result in personal injury and property damage



NOTE
provides important information which must be observed



TIP
contains practical information to assist you with your work



Words highlighted in colour indicate buttons or click paths for the corresponding program point, e.g. Examination, Resting ECG

Words highlighted...

02.1 General notes

Strict compliance with the safety instructions protects against personal injury and property damage during device operation. This Operating Manual is designed to accompany the product and must be kept ready to hand close to the device.

As either the operator or user of this device you should have read and understood this Operating Manual, in particular the safety instructions.

Laws and regulations applicable to the product

- This system is designed in accordance with Medical Device Directive 93/42/EEC, Class IIa, and meets the requirements of protection class I or II (depending on the power supply unit used; seca CT 321 meets the requirements of protection class III – device with an internal power supply), type BF (defibrillation protected), according to IEC 60601-1
- Other devices which are part of the system must meet the requirements of the Standard for Information Technology Equipment (IEC 60950) and the Standard for Electrical Medical Devices (IEC 60601-1).
- The electrical installations in the rooms in which the system is used must meet the requirements of the applicable safety standards.

02.2 Operator instructions and user requirements

As the operator you are responsible for:

- Ensuring that the operating personnel have been instructed in the technical use of the device and they have read the Operating Manual
- The device is used as intended by suitably trained specialist staff
- Compliance with the safety regulations and safety instructions, provisions for safety at work and accident prevention regulations
- Compliance with the maintenance instructions

Users must meet the following requirements:

- Knowledge of the correct use of the device
- Command of the required device handling processes
- Knowledge of the safety regulations applicable when using such devices
- Understanding of the potential hazards

The device may only be used by medical specialist staff, e.g. physician's assistants, nurses, assistant medical technicians, physicians, etc.

For users outside the Federal Republic of Germany, the respective national accident prevention measures, regulations and requirements apply.

02.3 Safety installations and safe working

seca CT 320/321 must only be used in a technically perfect condition. Regularly carry out a visual inspection of the device and the cables, in particular the patient cable. Only use accessories approved by seca.



The device is protected against defibrillation only in connection with the manufacturer's patient cable.

In the event of defibrillation, take note of the manufacturers' instructions regarding the safe and proper use of the defibrillator. Defibrillation has an interfering effect on the ECG recording. seca CT 320/321 has a recovery time of less than ten seconds.

Installation of the system

Portable socket outlets must not be laid on the ground.



Portable multiple socket outlets which are supplied with the system are to be used only for supplying devices which are part of the system. Additional portable multiple socket outlets, lines and other equipment, which are not part of the system, must not be connected to the system.

When using a multiple socket outlet, the maximum permitted load is 3200 VA.



Slots which are not used in the delivered system (portable multiple socket outlets) must be provided with covers.

Ambient conditions, handling

seca CT 320/321 is not suitable for use in rooms or areas with a risk of explosion.




Strong electromagnetic sources in the immediate vicinity of the seca CT 320/321 may result in recording errors (e.g. HF electrosurgical devices, mobile phones, induction furnaces).

The device must not be operated simultaneously with HF surgical equipment.

The device must be protected against the ingress of moisture, dust or dirt and also against mechanical impacts such as damage due to dropping or transit damage.

02 Safety instructions

Patient safety

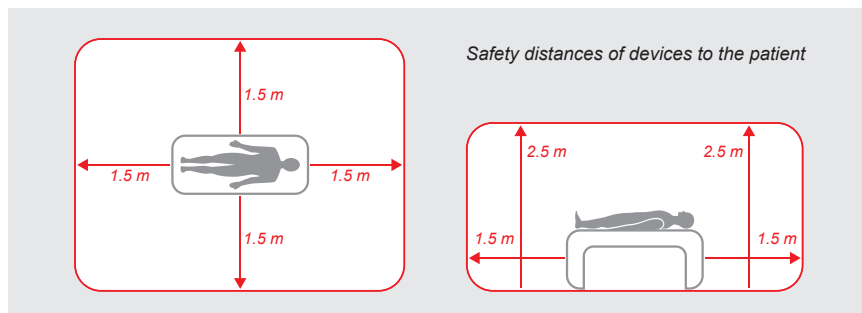

Without medical protective devices, for example medical protector, the PC and all the non-medical devices connected to the system (e.g. the monitor and printer) must be set up and used at a distance of at least 1.5 m to the patient unit (see the orange area in the figure) as leakage currents can occur.

When coupled with other devices, make sure that there is no danger due to the accumulation of leakage currents.

During routine maintenance work to non-medical devices connected to the system the patient must not be touched (risk of electric shock).

Make sure that the electrode contacts do not come into contact with other conductive parts.



All results of automatic analysis and the resulting unconfirmed reports produced by the system must be considered as suggestions only. For diagnosis and therapy purposes it is essential that the results are checked and assessed by a qualified physician.

seca CT 321 (Bluetooth), rechargeable lithium-ion battery



Remove the rechargeable battery if the device is not used for a long period.

Only use the supplied charger to charge the rechargeable battery.

Do not expose the rechargeable battery to extreme temperatures, fire and moisture. Do not immerse the rechargeable battery in liquids. Observe the operating and storage conditions.

Do not subject the rechargeable battery to strong shocks or drop it. The rechargeable battery must not be disassembled, modified or short circuited.

Hygiene

For cleaning and disinfection observe the legal requirements and the current state of technology.

Use only cleaning agents and disinfectants approved by seca for cleaning and disinfection. Clean and disinfect your device in accordance with the specifications given in *chapter 04 Hygiene*.



Display of the ECG signal in seca cardio 150

If electrodes become detached from the patient during an ECG recording or the electrode contact is too weak, a red signal line will be displayed on the corresponding ECG channel in seca cardio 150. Below the ECG recording a hint will appear (in red letters) indicating which electrodes are concerned. Reattach them.



The appearance of red signal lines in seca cardio 150 does not indicate that the patient has an asystole.

seca cardio 150 provides pacemaker detection. Here, the pacemaker pulse from the ECG signal (in two channels at least) is detected and then projected into the ECG recording as an (artificial) spike, precisely timed. However, the pulse width of the pacemaker is not calculated with the pacemaker detection in seca cardio 150.

The device is not suitable for binding pacemaker checks. In case of doubt, use the device approved by the pacemaker manufacturer (see the patient's pacemaker record).

System and data security



The device must only be used with the supplied seca software (seca cardio 150).

As the operator you are responsible for ensuring regular data backups (patient databases, evaluations etc.) and system backups. We recommend that you backup the data at the latest before new installations, updates and far-reaching system configurations.

seca cardio 150 new installations, updates and system configurations may only be performed by your authorised seca dealer.

Only change data generated in seca cardio 150 within seca cardio 150 itself and not in your EPR system (Electronic Patient Record) or your hospital information system (HIS).

seca does not accept any responsibility for any changes to data in your EPR system or your HIS which were made after the export from seca cardio 150.

To ensure the safe operation of seca cardio 150, deactivate the screen-saver and energy management options on your PC.

Set up your operating system in such a way to prevent the PC from being switched off either accidentally or automatically during the examination (standby mode/idle mode).

02.4 Information on EMC (Electromagnetic Compatibility)

The use of other accessories, other converters and leads than those indicated, except for the converters and leads sold by seca as spare parts for inner components, can lead to increased electromagnetic emissions or to a reduced electromagnetic immunity of the system. For connecting the device to other equipment, only specially screened cables supplied by seca must be used.

02.5 Maintenance (regular safety checks)

The operator is responsible for maintenance. The operator must ensure that the device is checked for proper condition at the latest every two years. The functionality and the state of accessories must be checked at regular intervals. If damaged and/or heavily soiled, the complete system must no longer be used.



All interventions in the existing system, changes to system components, enhancements as well as internal cleaning and repairs may only be performed by your authorised seca dealer or seca.

Technical safety check

After each system or device repair, modification or conversion, a technical safety check must be performed by your authorised seca dealer.

02.6 Disclaimer

The manufacturer is not responsible for improper operation, failure to comply with the safety instructions and non-observation of specifications due to negligence.

seca will only assume responsibility for the safety and reliability of the seca CT 320/321 when all changes, enhancements, repairs and other work to the device or system have been performed by an authorised seca dealer or seca and the Operating Manual has been observed during device operation.

02.7 Warranty

Our product philosophy is committed to providing you with faultless products which meet your expectations. Should you have reason to complain we aim to rectify any defects immediately or provide a replacement delivery.

This does not include damage that can be attributed to usual wear and tear, improper use, unauthorised modification of parts and the use of violent force.

After the expiration of the warranty period only use original spare parts and accessories from seca. This is the only way to ensure the safe and trouble-free operation of your device.

Warranty periods

seca CT 320/321.....	12 months
ECG electrodes	without warranty

03

Description of device, initial operation

03.1 Part names, components for ECG recording



seca CT 320 (with USB connectivity)



seca CT 321 (with Bluetooth connectivity)

Rechargeable lithium-ion battery, Charger with power supply unit,
Bluetooth USB stick

Note about the electrodes


For optimal results, we recommend using high-quality ECG electrodes and ECG preparation cream to prepare the electrode application points.

seca CT 320/321 – Length of patient cables

R, L, F, N.....approx. 1050 mm
C1 – C6..... approx. 700 mm

03 Description of device, initial operation

03.2 Operation of seca CT 321 (with Bluetooth connectivity)

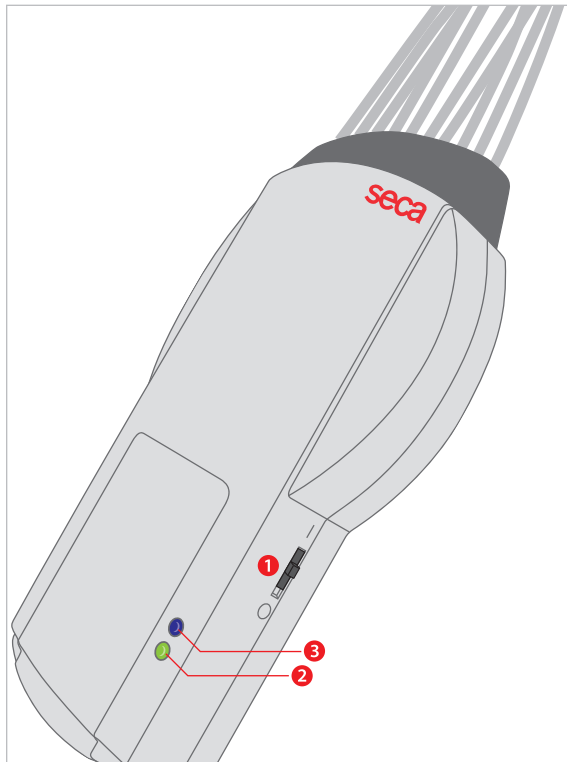


On/off switch

The switch on the right side of the housing ① switches the device on and off. Only switch on the device immediately before the start of recording and switch it off again directly after the recording is completed. The maximum operation duration is 4 hours after which the rechargeable battery will need to be replaced or charged.

When the device is switched on, the lower LED ② indicates the charge status of the battery:

Green:	100 % – 65 %
Orange:	65 % – 35 %
Red:	< 35 %


If the lower LED flashes red after switch-on, the rechargeable battery must be immediately removed from the device and charged.

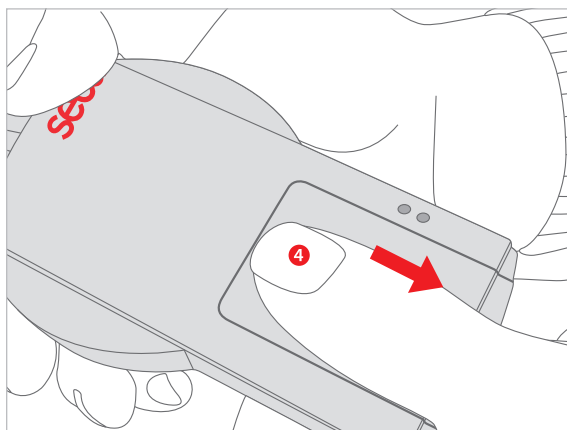

The upper LED ③ indicates whether the Bluetooth connection is active between the device and the PC. At the start of an ECG recording, the Bluetooth connection is established between the device and the PC and the upper LED lights blue. The precondition for this is that the Bluetooth connection has been installed correctly (see 03.4 Connecting the ECG device to the PC). If no connection has been established, the LED will not light.

Note on the rechargeable battery

The rechargeable lithium-ion battery does not have memory effect. The battery can be charged at any time, irrespective of the discharging times.

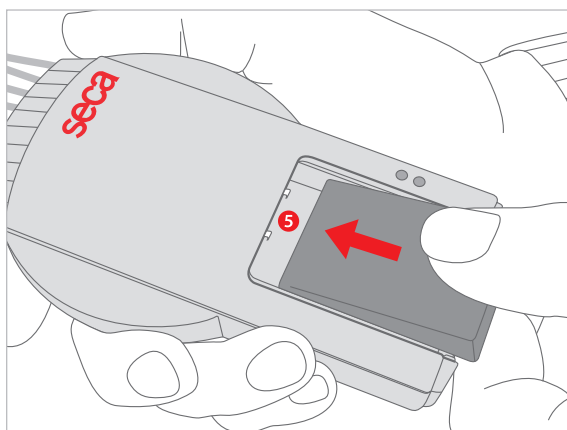


Charging the rechargeable battery

With your thumb, slide the battery cover back ④ and take out the battery. Insert the battery into the supplied charger. A description of the charging method can be found in the Operating Manual of the charger. The charging time is 2 to 3 hours. Reinsert the charged battery so that the contacts of the battery and seca CT 321 come into contact ⑤. Close the battery cover.

03.3 seca cardio 150 – basic program structure

The program is divided into three areas – **User**, **Patient** and **Examination**. This structure ensures that you can always recognize who (which user) is carrying out what type of examination with whom (which patient). The main menus of each area can be reached by clicking on **User**, **Patient** or **Examination**.

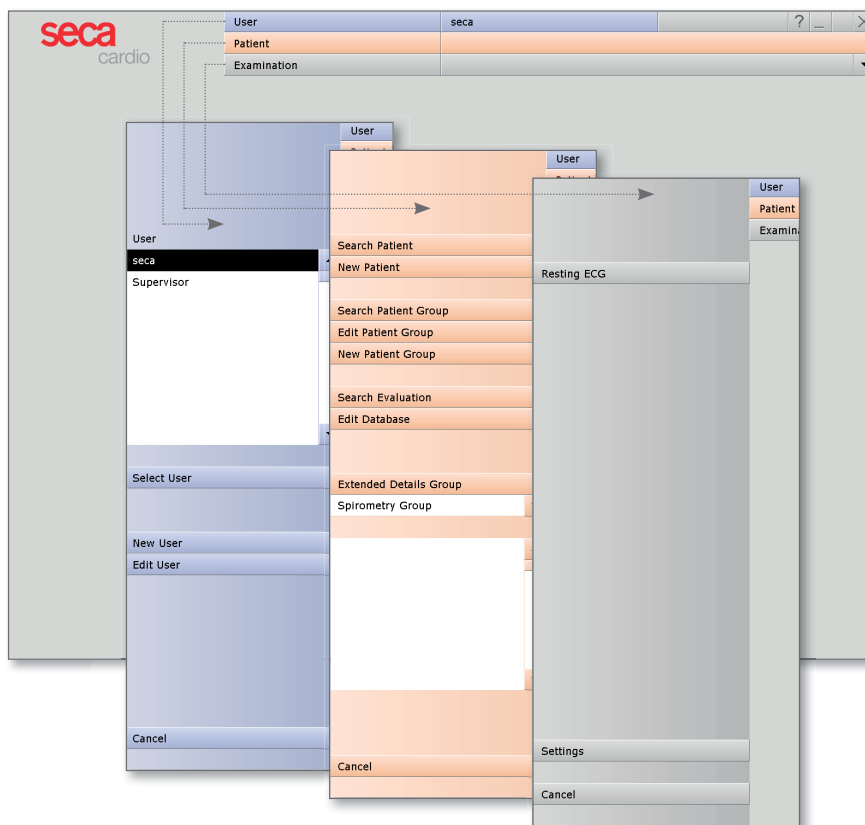

In the **User** main menu the users of the system can be created and managed. The user administration can be used to allocate user rights and control user-specific settings, e.g. the creation of a separate patient database for each user.

The **Patient** main menu is used for patient management. Its most important functions include **Search for Patient**, **New Patient** and **Search Evaluation**.

The **Examination** main menu is linked to the **Settings** area. This area is for making cross-program, examination-related and user-specific settings.

03 Description of device, initial operation

03.4 Connecting the ECG device to the PC

Precondition:

seca cardio 150 is installed on your PC and ready for operation.

The seca devices and components may only be connected to the PC after seca cardio 150 has been installed. The required device drivers are installed using the seca cardio 150 standard setup or via targeted selection during the seca cardio 150 setup.

03.4.1 ECG device with USB port

➤ Connecting the USB cable to the PC


Connect the USB cable of the ECG device to the PC. Power is supplied via the USB cable. The Windows driver installation for the new hardware starts automatically. Select the “Find driver automatically” option. Use the **Next** and **Finish** buttons to control the installation process.

➤ Setting up the interface in seca cardio 150

Start seca cardio 150. Select **Examination**, **Settings**, **Connection**, **Device** ①. Select **ECG**, **seca CT 32x** ② in the device selection list; select **USB** in the interface selection list ③. Click on the **Save** button (at bottom left).

➤ Selecting ECG device for resting ECG

See section *03.4.3 Selecting ECG device for resting ECG*.

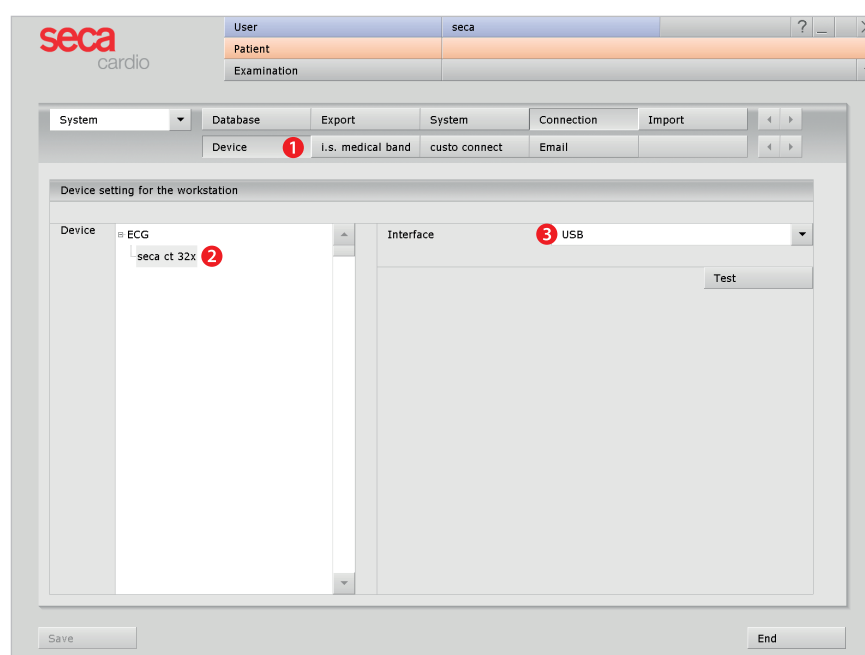

03.4.2 ECG device with Bluetooth connection

Note on Bluetooth USB stick

The supplied Bluetooth USB stick is supported by Windows XP and Windows 7, but not by Windows 2000 and Windows Vista.



To ensure the smooth installation of the supplied Bluetooth USB sticks in **WINDOWS XP**, the full installation of the current seca cardio 150 version (not the update) must be carried out from the CD, even if another version of seca cardio 150 is already installed. Otherwise, the operating system will need to be accessed manually. The seca cardio 150 installation must be completed before connecting the Bluetooth USB stick to the PC.

For **WINDOWS 7**, no subsequent installation of seca cardio 150 or additional access is required as the Bluetooth USB stick is already known in the operating system.

Overview Bluetooth installation

1. Connect the Bluetooth USB stick to the PC
2. Driver installation for the Bluetooth USB stick
3. Check the driver installation
4. Install the Bluetooth interface in seca cardio 150
5. Allocate the device for examination
(see 03.4.3 Device settings for resting ECG)

A description of work steps 1. to 3. can be found in the “Quick guide for installing Bluetooth devices in seca cardio 150”. This Quick Guide is located on your seca cardio 150 CD.


IMPORTANT:

Before continuing with the configuration of the Bluetooth interface in seca cardio 150, make sure that the Bluetooth USB stick is correctly connected and the power supply to the ECG device is guaranteed.

03 Description of device, initial operation


Installing the BT interface in seca cardio 150

Start seca cardio 150. Select **Examination**, **Settings**, **Connection**, **Device** **1**. Select **ECG**, **seca CT 32x** **2** in the device selection list and **Autom. BT-COM search** **3** in the interface selection list. Click on the **Save** button (at bottom left).

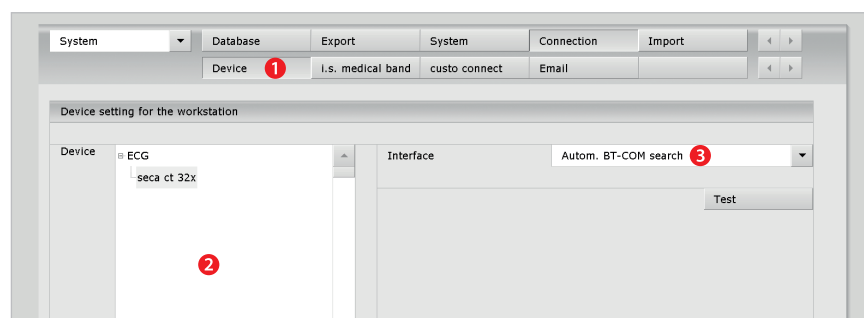

Then select **Examination**, **Settings**, **Connection**, **i.s. medical band** **4**. Activate the **Microsoft Stack** **5** option, click on **Save** **6** and then on **Open** (to the left of Microsoft Stack). The message "Interface is enabled" appears **7**.


Switch on seca CT 321 and click on the **Search** button **8**. The "i.s. medical band search" dialogue appears. Restart the search if the device is not found (repeat the process several times if necessary). Select the desired device and click on **Confirm**. The previously selected device is displayed in the table on the i.s. medical band page **9**. **Save** **6** your input.

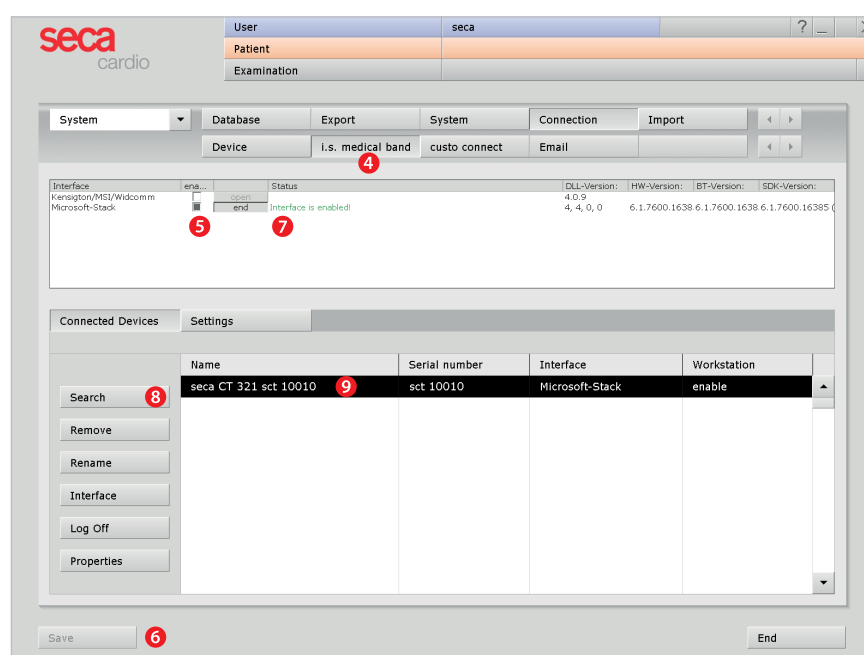

To establish the Bluetooth connection, double-click on the device name in the table **10**. In the open dialogue, go to the **Connection** **11** tab and click on **Connect** **12**. After the connection has been established, the message “Serial connection is enabled! End” will appear. **End** **13** the dialogue.


If the Bluetooth connection was successful between the PC and seca CT 321, the LED on the device will light blue.

Save your input and close the i.s. medical band page with **End**. Restart the program to transfer the changed parameters to the configuration database.

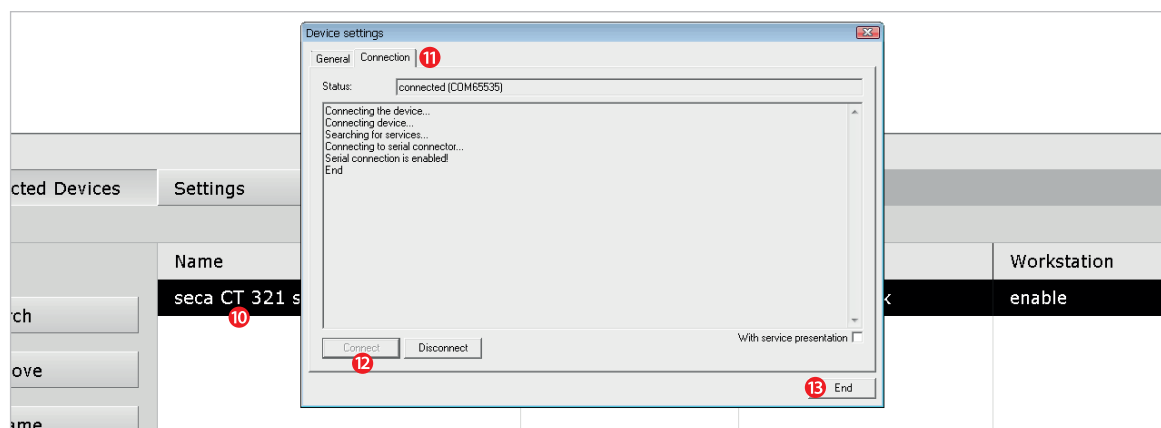

03.4.3 Selecting ECG device for Resting ECG

- In seca cardio 150, select: **Examination**, **Resting ECG**, **Settings**, **Connection** **1**, **Device** **2**.
- Select **seca CT 32x** **3** from the list under “ECG Device”.
- Click on the **Save** button (at bottom left) to apply your input.
- Close the page with **End** (button at bottom right).
- The device is ready for operation.

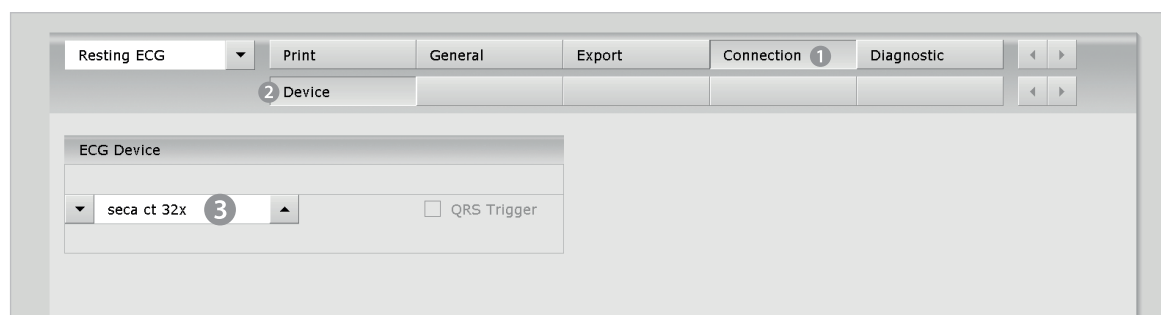

04.1 Cleaning and disinfection

Important notes



Use only cleaning agents and disinfectants recommended by seca. Unsuitable agents may damage the device.

Observe the manufacturer's specifications (e.g. regarding dosage and contact times).

The device must never be immersed in liquids or cleaned with too much water.

The device must not be connected to a power source during cleaning and disinfection.

Surface cleaning

Make sure that the exterior of the device is always aesthetic and clean. Wipe the device using a damp cleaning cloth and a mild (acid-free!) cleaning agent or a suitable disinfectant.

Cleaning agents and disinfectants must not be sprayed directly on or into the device.

04.2 Recommended cleaning agents and disinfectants

Disinfectants:

All alcohol-based disinfectants (e.g. propanol, ethanol)
approved for medical use.


05

Carrying out an examination

05.1 Preparing the patient, examination procedure

- Make sure that the ECG device is connected to the PC and the power supply to the device is guaranteed.
- Check that your patient is lying comfortably and is not cold.
- Shave, clean and dry the electrode application points thoroughly.
- Place the electrodes on the patient.
- Connect the patient leads to the electrodes.
- Start seca cardio 150 and click on:
Examination, Resting ECG, New Resting ECG,
select the patient for the examination and start the recording
(a precise description can be found in *Chapter 05.2*).
- The patient should not move or talk during the recording process.

05.2 Electrode application

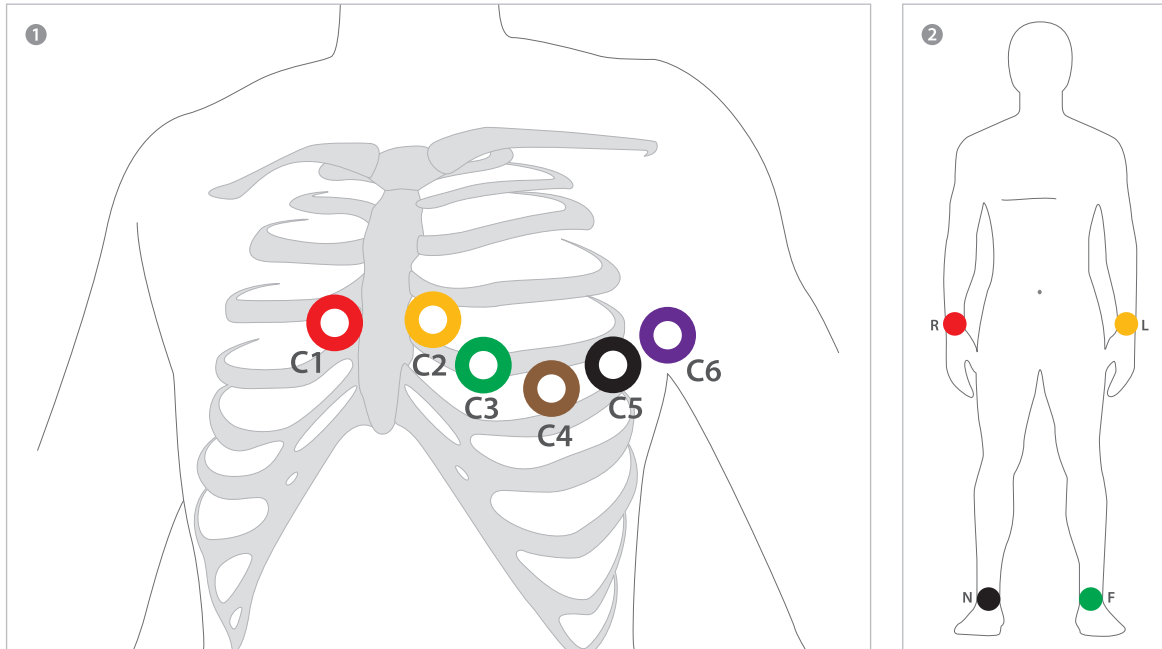

1 Chest wall, standard according to Wilson


V1 (C1)		red	4. Intercostal space at the right sternal border
V2 (C2)		yellow	4. Intercostal space at the left sternal border
V3 (C3)		green	On the left on the 5th rib between C2 and C4
V4 (C4)		brown	5. Intercostal space on the left midclavicular line
V5 (C5)		black	On the left on the anterior axillary line, on the level of C4
V6 (C6)		purple	On the left on the midaxillary, on the level of C4

2 Extremities

R		red	Right arm
L		yellow	Left arm
F		green	Left leg
N		black	Right leg

05 Carrying out an examination

05.3 First work steps in seca cardio 150

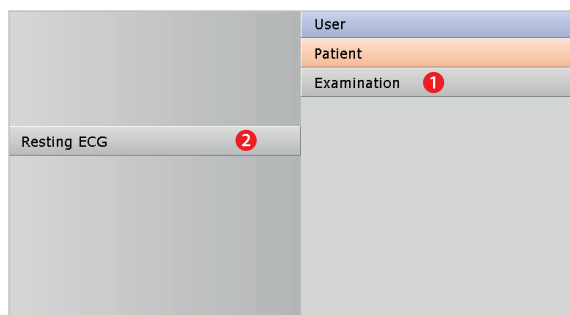


Note on the procedure

The steps necessary to record and evaluate ECG data in seca cardio 150 are shown without a surgery IT system or HIS connection.

Program start, calling the ECG

Start seca cardio 150 and log on with your user name and password if required.

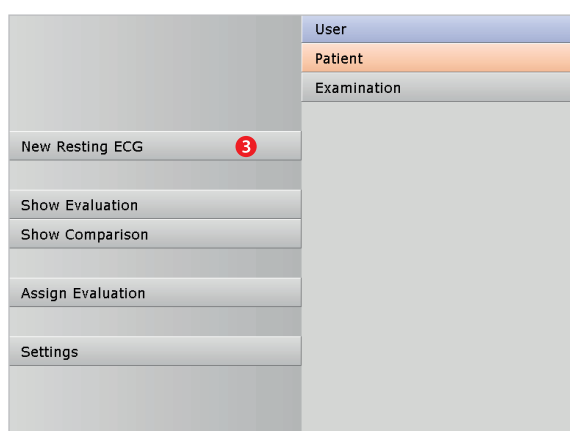


Click on: Examination 1, Resting ECG 2, New Resting ECG 3.

Selecting the patient

Select a patient for the examination. Enter the patient's name or the first letter of their name into the input fields of the patient search screen ⁴.

Select the patient from the list below the input fields 5 and confirm your selection by clicking the **Select Patient** 6 button. You can also select the patient by double clicking on the corresponding name.

New patient

If the patient is not in your database, click on the **New patient**  button. Enter the patient data. The fields marked with an asterisk are mandatory.

Save the details to enter the patient in your database.

	User
	Patient
	Examination
Last name	Muster 4
First name	
Patient ID	
Patient Group	All patients
Assignment Physician	
Physician ID	

Last name	First name	Date of birth	Pat. ID
Mustermann	Franz 5	10.10.1960	

Select Patient 6
Edit Patient
New Patient 7
Cancel

05 Carrying out an examination

05.4 Recording resting ECG

05.4.1 Monitoring and electrode control

The patient's ECG signal will be shown on the display but not recorded (monitoring). Work steps before the start:

- Choose the type of lead ❶ (standard, left, right, Nehb, Cabrera).
- Check if all electrodes are attached optimally. If there are red lines ❷ on the screen, the contact between the skin and electrode(s) is insufficient and the corresponding electrodes need to be reattached. For ECG devices with USB connection, the **Electrodes** button ❸ provides you with a graphic representation of the signal quality of all electrodes.
- Set the required filters (**Options** menu ❹).



The preset standard procedures for automatic ECG and manual recordings are described in this chapter.

These procedures can be changed in the **seca cardio 150** settings:

1) Settings for Auto-start recordings or automatic ECG
In **seca cardio 150**, select **Examination, Resting ECG, Settings, General, Workflow**. The recording duration and procedures after the recording can be set in the "Automatic ECG" area. Click on **Save** to apply changes.

Print settings for automatic ECG:
The contents for a printout after an automatic ECG recording can be compiled at **Examination, Resting ECG, Settings, Print, Automatic** in the "Printout Automatic ECG" area.

This setting is only required if the automatic ECG print pages should contain content other than the data already defined in the standard print settings for resting ECG (see **Examination, Resting ECG, Settings, Print, Default**, "Printout" area).

2) Settings for manual recordings
Under **Examination, Resting ECG, Settings, General, Workflow** in the **Workflow** area (in the first block "After recording") the procedures after the recording can be set. Click on **Save** to apply changes.

Note on the recording duration:
At least 10 seconds of the ECG signal must be recorded. A recording cannot be ended before this time has elapsed.



05.4.2 Starting and ending a resting ECG

Automatic ECG (Auto Start button ❶):

Click on **Auto Start** ❶ to start the automatic recording. The standard setting for the duration of automatic recording is ten seconds. After the ten seconds have elapsed the recording is automatically ended, saved, measured and printed out¹⁾.

Manual recording (Start button ❷ or Enter key):

If you want to perform a recording without a time limitation (e.g. because of suspected irregularities), you have to activate the recording with **Start** ❷. Click on **End** ❸ to close the recording. Click on **Confirm** in the End dialogue to ensure that the recording is saved, measured and displayed as an evaluation²⁾.

05.4.3 Editing options during the recording process

Marking ECG sections

Automatic marking: By clicking on the **Mark** button ①, the last six seconds are automatically marked. A dialogue appears for specifying³⁾, saving and printing the marked part ③.



Manual marking: By clicking on the **Start Mark** button ② you can determine the length of the marking yourself. The marking continues to run until you click on the **Stop Mark** button. A dialogue appears for specifying³⁾, printing and saving the marked part ③.



Tip:
Keyboard shortcut



Change amplitude
(mm/mV)

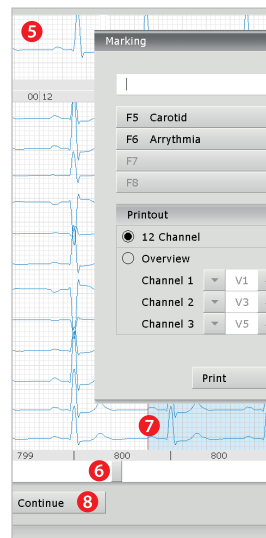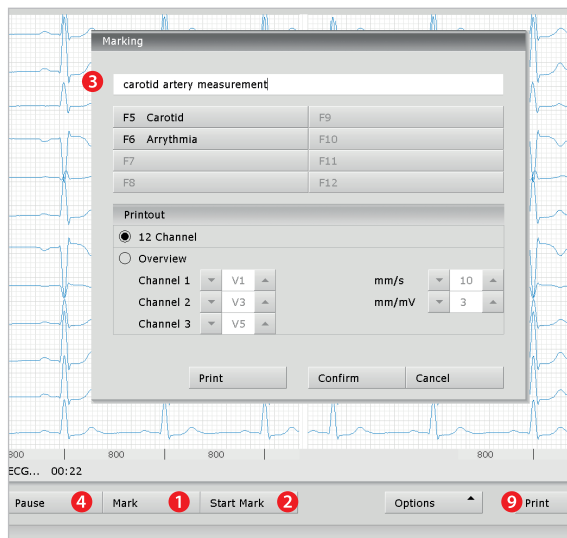

3) **TIP:** Text modules for specifying marked parts

Use: If a part is marked during the recording, the "Marking" dialogue appears. The marked parts will be automatically specified by pressing a previously configured text module button (e.g. F5).

Configuring text modules: Under **Examination, Resting ECG, Settings, General, Marking** the text modules can be configured for specifying marked parts. A name is assigned to the text module in the "Name" field. This name later appears on the button for calling the text module. In the "Text module" field, enter the text which will later be used to specify the marked parts. **Save** your input.

Viewing and marking ECG, measuring HF during a pause

Clicking on **Pause** ④, will stop the screen display. The recording continues to run and is displayed on one channel ⑤ (fig. right). The scroll bar ⑥ can be used to scroll through the current recording.

The tools **Mark**, **Measure HR** and **Calliper** can be found at the top right in the Mouse menu. By dragging the red cursor ⑦ in the ECG, you can mark ECG sections (using the **Mark** mouse function). A dialogue appears for specifying, printing and saving the marked part. With **Continue** ⑧ you return to the normal view of the recording.

05.4.4 Online ECG printing

Click on **Print** ⑨. A screen page of the ECG signal is printed from the point of clicking. The printout contains 4.5 to 9 sec of the ECG signal, depending on the display speed⁴⁾.

4) **Note on online printing:** Under **Examination, Resting ECG, Settings, Print, Default**, you can define in the "Online ECG printout" area whether the ECG should be printed as it appears on the monitor or if online printing should be carried out according to previously defined print settings for analysed ECG.

06 Working with the evaluation

06.1 Opening the evaluation

seca cardio 150 offers different options to open an evaluation, e.g. via the evaluation search or the main menu of the respective examination.

Opening an evaluation via the evaluation search

Click with the **right mouse button** on the **Patient** **1** button. This opens the evaluation search.

In the Examinations area, enter what type of evaluation you are searching for, e.g. **Resting ECG** **2**. In the Properties area **3** you can define more search criteria.

*If you set the **confirmed** preference to **No**, you will receive a list of all the evaluations which have not yet been confirmed – a type of to-do list.*

To start the search, click on **Search Evaluation** **4** or activate **Search automatically** option **5**. This option triggers an automatic search in your database whenever the search criteria are changed.

The right part of the screen displays a list of all the evaluations which correspond to the activated search criteria. To open the desired evaluation, select it from the list and click on the **Show Evaluation** button **6** or double-click on the evaluation.

If you want to use the same search criteria for the next search, activate the **Save Selection** **7** option.



*Reference between
End dialogue
and search screen*

In order to make proper use of the search screen, the correct examination status must be defined in the End dialogue when closing an examination.

Example: An examination can only be found in the search screen with the preference “confirmed: No” when the evaluation status “Confirmed” is NOT selected in the End dialogue.

The screenshot shows the evaluation search window. At the top, there are buttons for 'Patient' (1) and 'Examination'. Below this is the 'Examinations' section with a search filter 'Resting ECG' (2). The 'Properties' section (3) contains various checkboxes for search criteria, including 'confirmed (B)' which is set to 'No'. The 'Filter properties' section includes 'Period' (all), 'Physician', 'Department', and 'Social security'. At the bottom, there are buttons for 'Search Evaluation' (4), 'Cancel', 'Save selection' (7), and 'Search automatically' (5). On the right, a table lists evaluations with columns for 'Exam.', 'Date', 'Patient', and 'Prop.'. The table shows two entries for 'Resting ECG' on '29.02.2012' for 'Mustermann Franz'. At the bottom right, there are buttons for 'Show Evaluation' (6), 'Print Evaluation List', and 'Export List of Evaluations'.

Exam.	Date	Patient	Prop.
Resting ECG	29.02.2012	Mustermann Franz	-----
Resting ECG	29.02.2012	Mustermann Franz	-----

Opening an evaluation via the examination main menu

Open the examination main menu via **Examination**, **Resting ECG** and then click on **Show Evaluation** ①.

New Resting ECG

Show Evaluation ①

Show Comparison

Assign Evaluation

Settings

Cancel

User

Patient

Examination

Last name

First name

Patient ID

Muster ②

Patient Group

Assignment

Physician

Physician ID

All patients

Last name

First name

Date of birth

Pat. ID

Mustermann

Franz ③

10.10.1960

Select Patient ④

Edit Patient

New Patient

Cancel

The patient search screen appears. In this screen select the patient whose evaluation you want to open. Enter the patient's name or the first letter of their name into the input fields of the patient search screen ②.


Select the patient from the list below the input fields ③ and confirm your selection by clicking the **Select Patient** ④ button. You can also select the patient by double-clicking on the corresponding name.

A list containing all the evaluations of the patient is then displayed. Select the desired evaluation from the list and open it by means of a double-click or via the **Show Evaluation** button.

06.2 Resting ECG evaluation


06.2.1 Screen pages within an evaluation

The ECG evaluation is divided into two main areas, **ECG** and **Measurement** (the **ECG** page is preset as the start page of the evaluation). From the subpages of the two areas the main page of the other area can be accessed at any time.

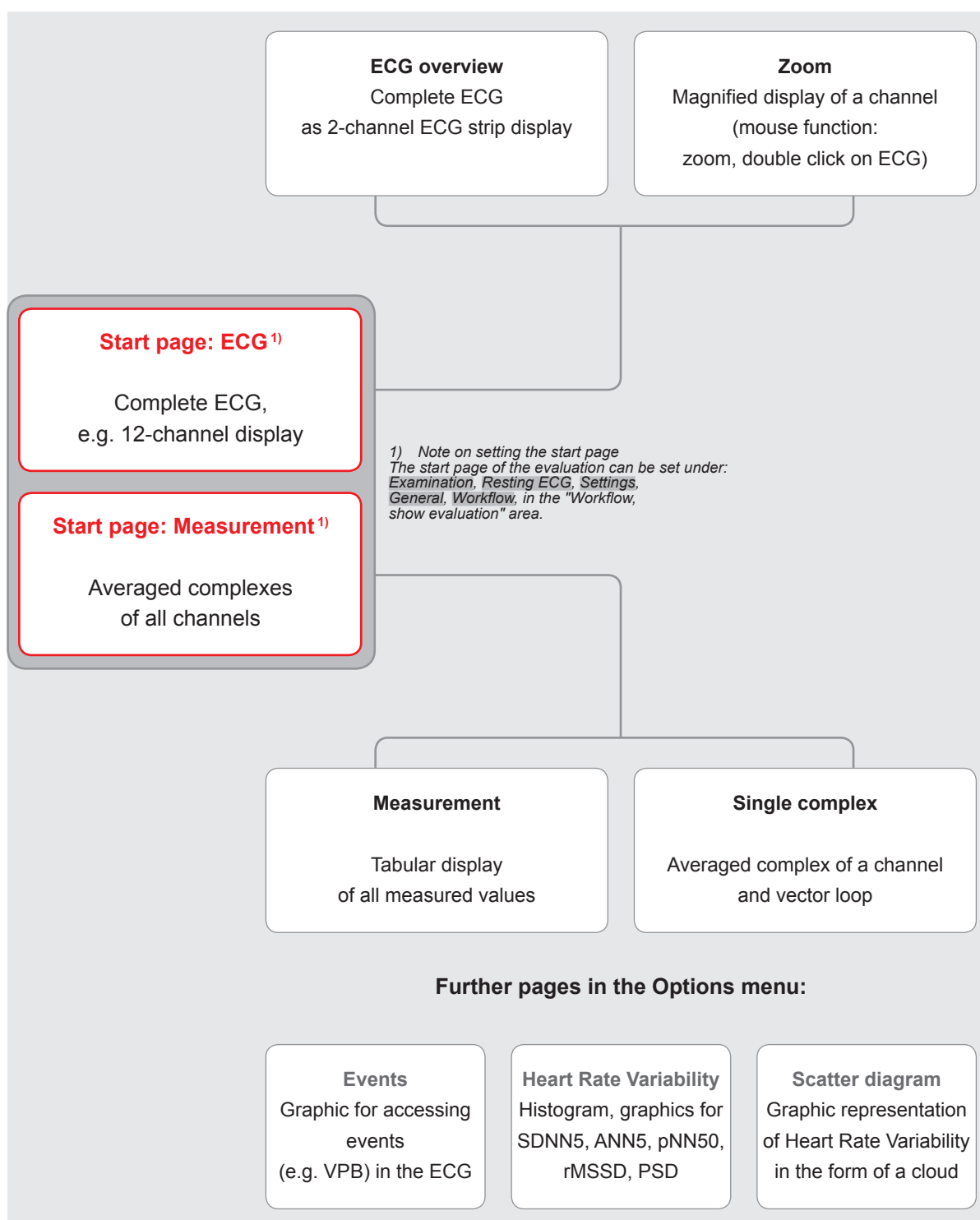

06.2.2 Comparing several evaluations from one patient


The current evaluation can be compared to other evaluations of the patient using the **Comparison** button. The comparison pages can only be opened if additional ECG evaluations from the patient are available. The **Evaluation** button will bring you back to the single view.

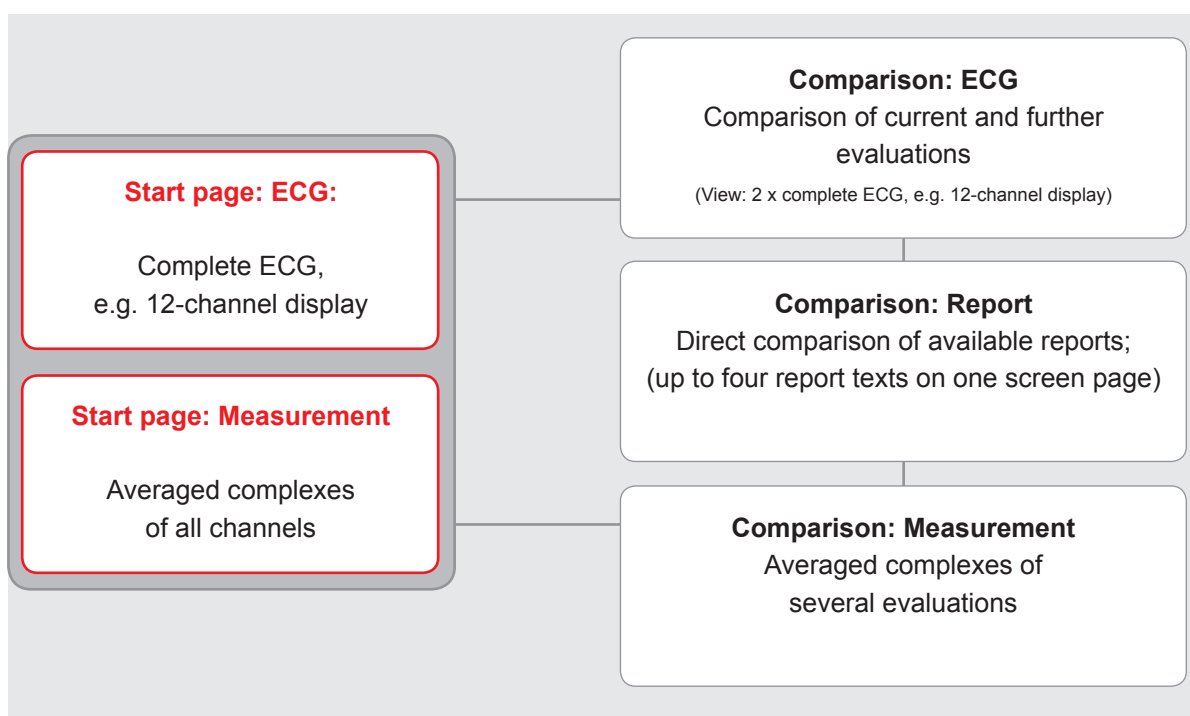

06.2.3 Note on navigation in the evaluation


The buttons for opening further evaluation pages are located on the lower edge of the screen. The labelling of the buttons changes as soon as you switch to a different evaluation page. The button that has been clicked always contains the name of the page you just left.

Example: You click on the **Measurement** button in the evaluation (view: ECG start page). You go to the the Measurement screen and the previously clicked **Measurement** button changes to **ECG**. By clicking on **ECG**, you can return to the previous ECG viewing mode.

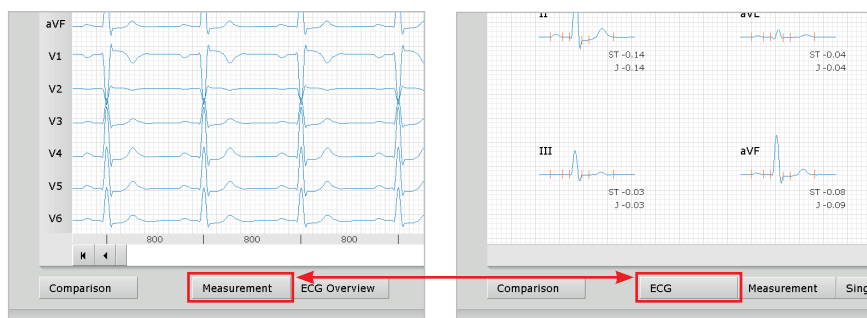

06 Working with the evaluation

06.2.4 Overview of evaluation pages

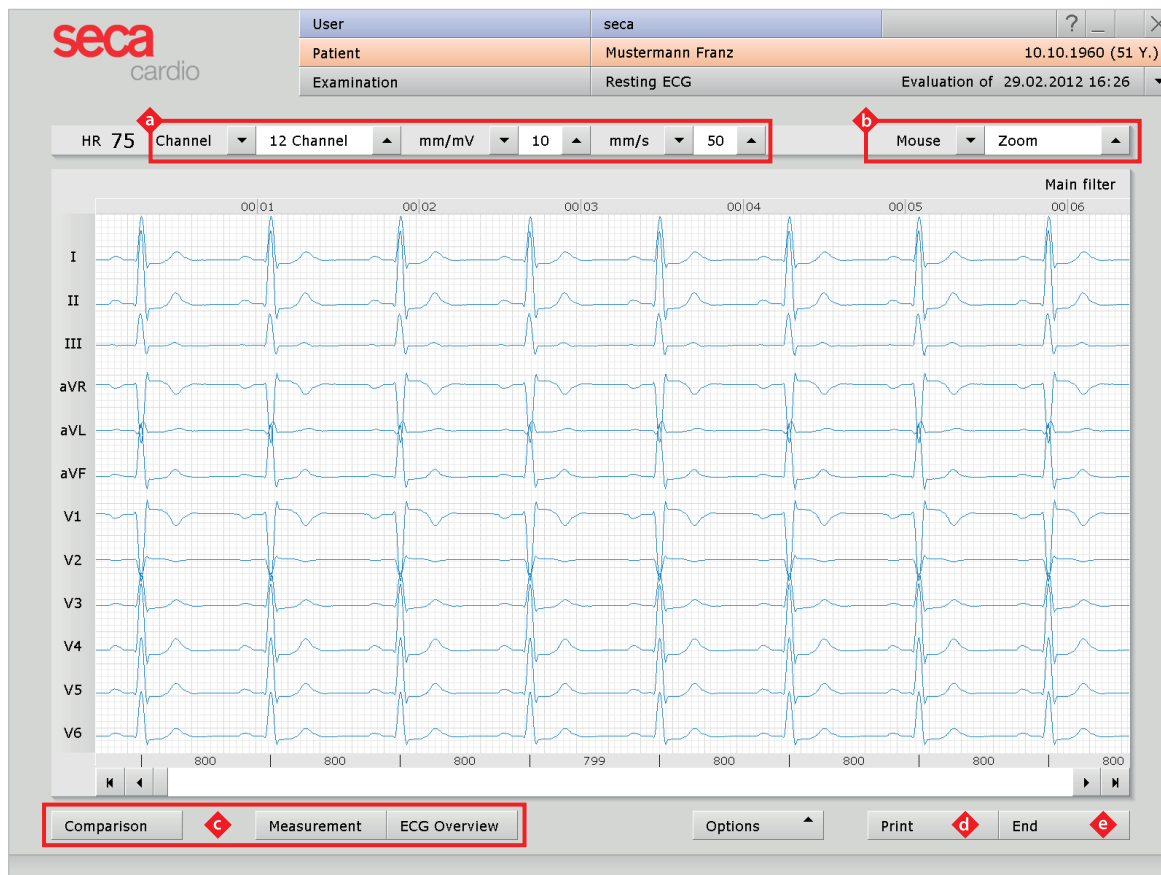

Elements on the ECG start page

2) Settings for the **Print** button:

Under **Examination, Resting ECG, Settings, General, Workflow** in the "Workflow, Default" area, you can define whether the extended print menu should be displayed when the **Print** button is clicked (standard setting) or whether printing should be conducted automatically according to the standard print settings (= option **Default printout**) without any additional settings being made.

The standard print setting for resting ECG can be found under **Examination, Resting ECG, Settings, Print, Default**.


Click on **Save** to apply your changes.

- a** Options for ECG display
- b** Mouse functions for detailed viewing and measuring of the ECG signal (Zoom, Analysis, Measure HR, Calliper, Mark)
- c** Further evaluation pages
- d** Button for printing the evaluation ²⁾
- e** Button for closing the evaluation

If the **Measurement** view is set as the start page, the same operating and navigation elements can be found there (**a** to **e**).

The start page of the evaluation can be set under:

Examination, Resting ECG, Settings, General, Workflow in the "Workflow, Show evaluation" area.

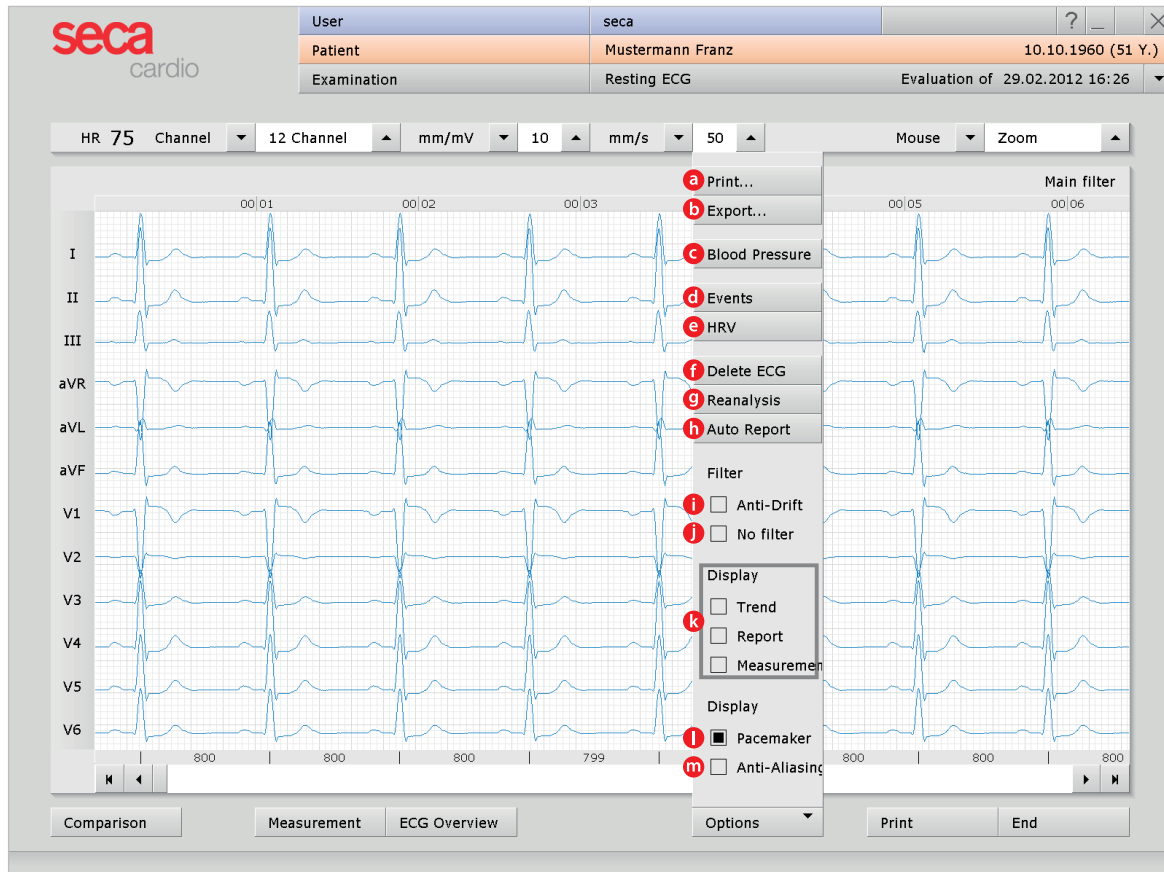

Content in the **Options** menu ³⁾

- a Print menu for temporary changes to the print settings
- b Button for exporting the evaluation (e.g. Excel, PDF, e-mail, DICOM)
- c Manual blood pressure input ("F9" key)
- d Trend graphic for displaying events in the ECG (e.g. VPB)
- e Tables and graphics for Heart Rate Variability⁴⁾ (e.g. SDNN5, ANN5, pNN50, rMSSD, PSD)
- f Deletion of ECG outside the marked sections
- g New analysis of ECG signal for resetting manual changes in the ECG, additions to the report remain
- h Automatic creation of a new report after manual changes have been made in the ECG recording
- i Anti-drift filter for compensating fluctuations in the zero line
- j Deactivation of all filters (display of raw data)
- k Fade in and fade out of additional content in the right half of the screen: e.g. HR curve and ST trend, averaged complexes and report or measurement table
- l Fade in and out of pacemaker spikes
- m Graphic flattening of ECG signal

3) Note on Options menu:
The scope and content of the **Options** menu changes based on which page of the evaluation you are currently on;

e. g. on the **Measurement** page in the **Options** menu, you can activate the display of **ST values** and set which marks should be displayed in the averaged complexes.


4) Note on Heart Rate Variability:
For the heart rate variability to be displayed, at least five minutes of ECG need to be recorded!

06 Working with the evaluation

Evaluation pages in the ECG area



Extended **ECG start page** with additional content.
In the **Options** menu, select **Report** to show the additional contents.
Measured values (**Measurement**), HR curve and (ST) **Trends** can also be displayed instead of the report.

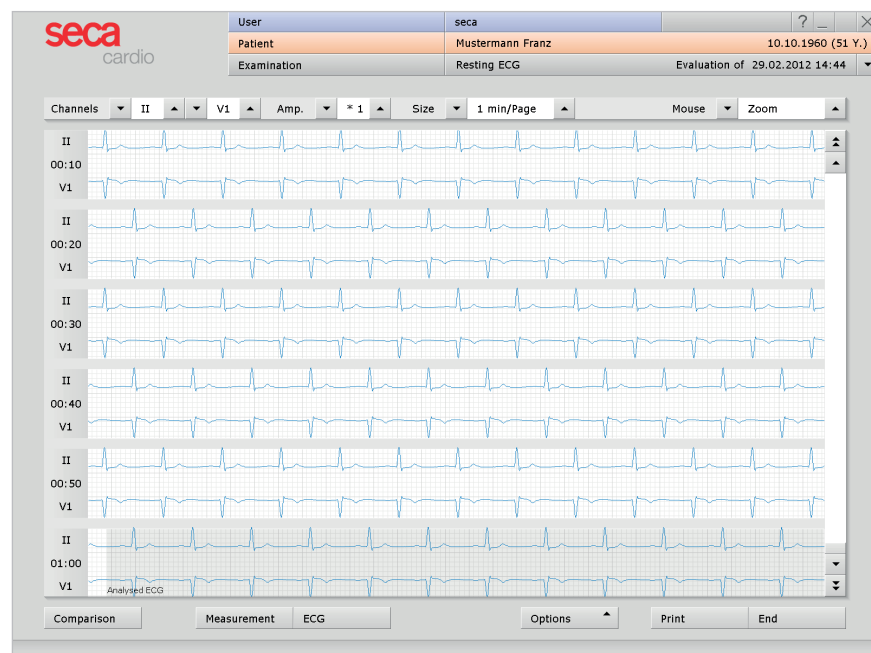

Select: **ECG Evaluation** > **ECG Overview**
Overview of complete ECG as 2-channel ECG strip display.



Select: **ECG evaluation**, mouse function **Zoom**, **double click on ECG**
Magnified display of a channel. By double clicking on the ECG again,
all or the set channels are displayed again.




Mouse functions:
Measure HR

The selection list "Mouse" (top right, above the ECG) contains, besides **Zoom**, the **Analysis**, **Measure HR**, **Calliper** and **Mark** mouse functions.

06 Working with the evaluation


Evaluation pages in the Measurement area

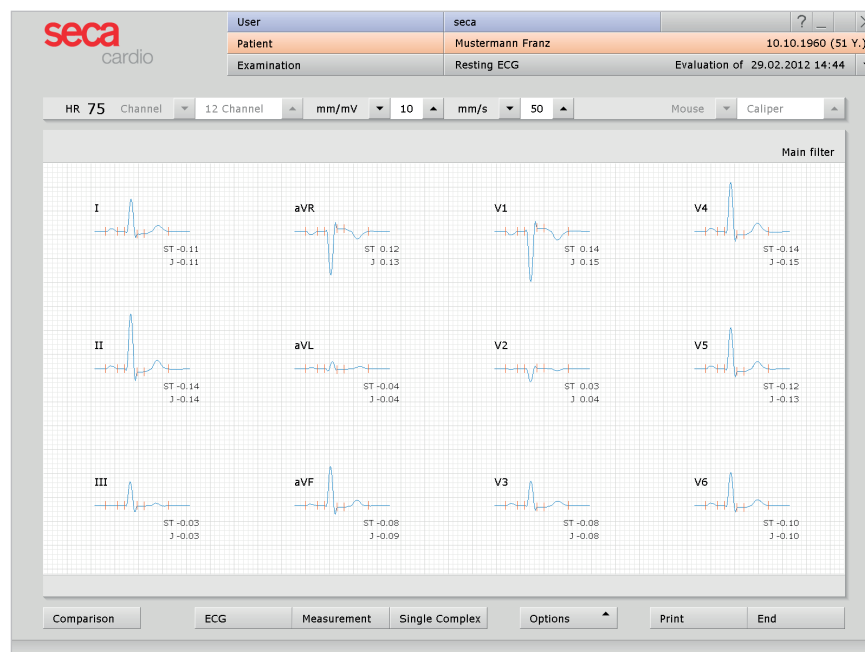

Select: **ECG Evaluation** > **Measurement**


Averaged complexes of all channels. In the **Options** menu, you can display the **ST values** for the averaged complexes.

Under Examination, Resting ECG, Settings, Diagnostic, Calculation can be set if the QTc duration is to be calculated according to Bazett (preset) or Fridericia. Save your input.

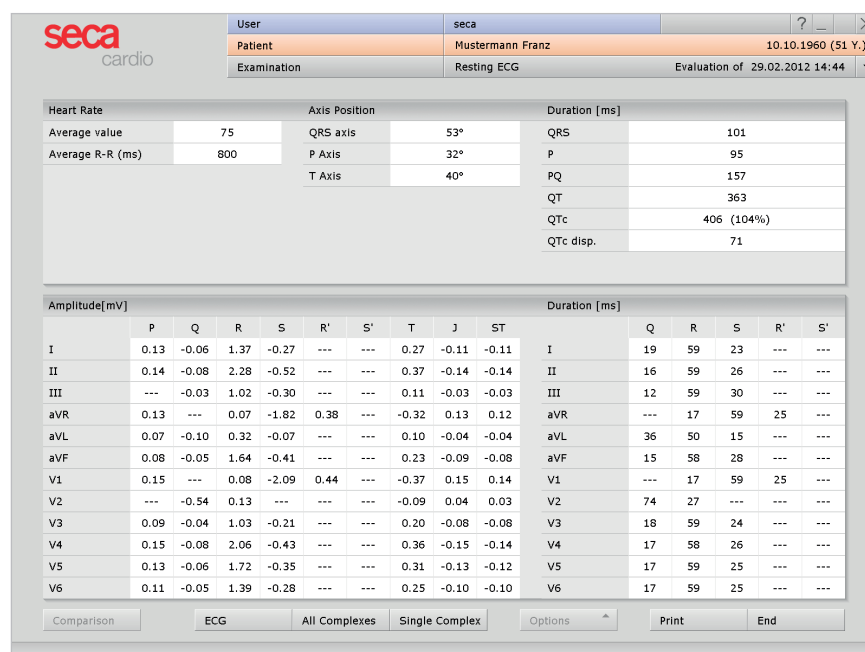

Select: **ECG Evaluation** > **Measurement** > **Measurement**

Tabular display of all measured values.

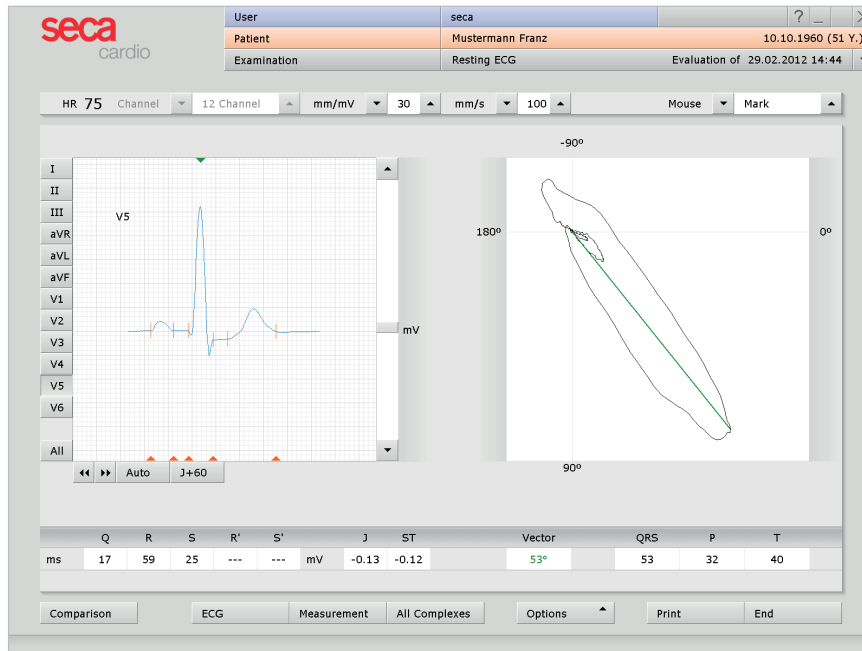

Using the **All** button, all averaged complexes will be shown on top of each other, highlighted light grey.

Select: **ECG Evaluation** > **Measurement** > **Single Complex**
Averaged complex of a channel and vector loop.


Further evaluation pages from the **Options** menu




Select: **ECG evaluation** > **Options** > **Events**

If the red cursor in the top graphic is moved over an event mark (= orange line), the corresponding ECG section will be displayed.

06 Working with the evaluation

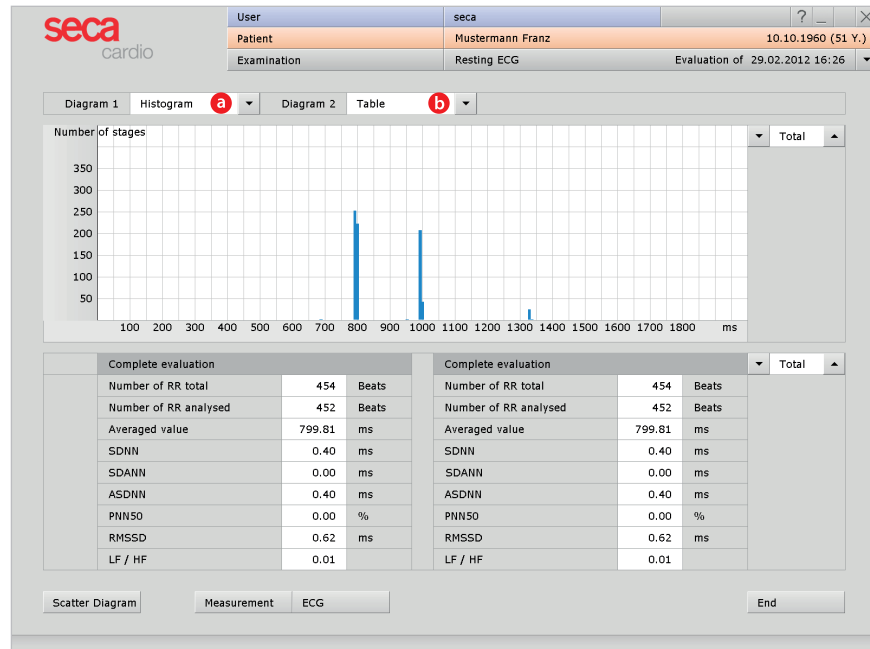

Select: **ECG evaluation** > **Options** > **HRV** (Heart Rate Variability)
Using the Diagram 1 **a** and Diagram 2 **b** menus, additional graphics for Heart Rate Variability can be opened.

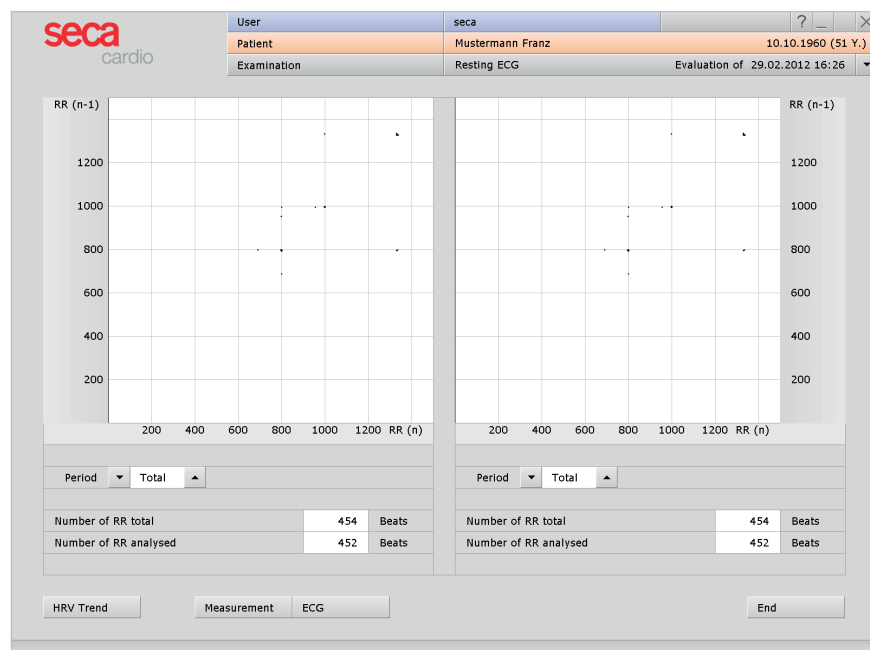

Select: **ECG evaluation** > **Options** > **HRV** > **Scatter diagram**
The more regular the heart rate, the fewer the points which differ from the diagonal of the diagram.


Comparison pages (comparison of different evaluations)



The **Comparison** button is deactivated (light grey) if no further evaluations of the patient are available.

Select: **ECG Evaluation** > **Comparison** > **ECG**

The current evaluation is compared with a further evaluation;
the selection lists above the ECG offer further evaluations for comparison.
With the **Report** button, the reports can be directly compared to each other.

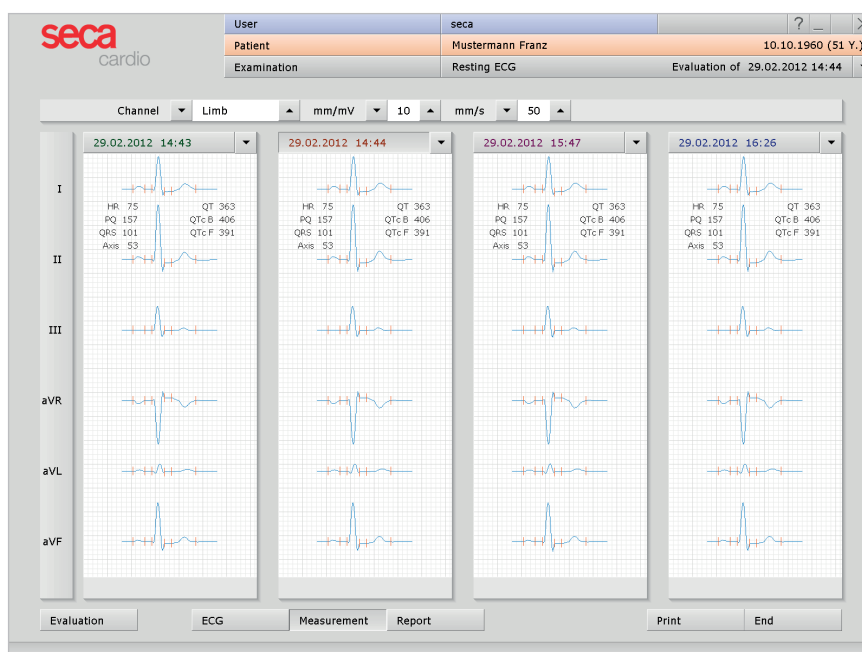


TIP: additional display of measured values

The display of measured values in the averaged complexes can be activated in the settings. This setting is available under **Examination, Resting ECG, Settings, General, ECG**. In the "Default settings for ECG" area, select the **AVG complex comparison with measured values** option.

Select: **ECG evaluation** > **Comparison** > **Measurement**

Comparison of averaged complexes of several evaluations


06.3 Writing the report

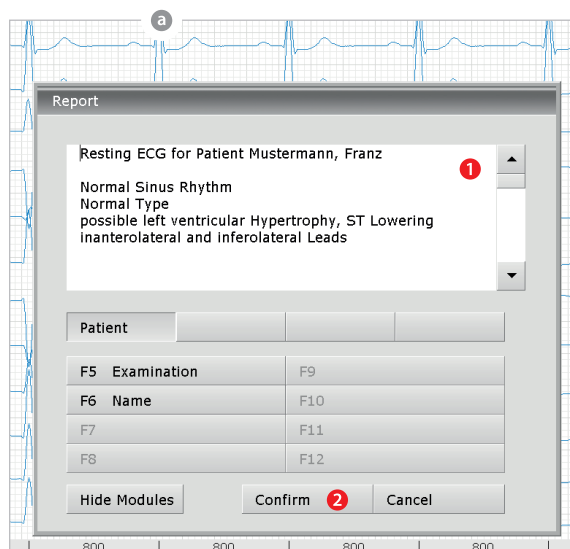

a The report is opened by right-clicking on the evaluation interface. In the context menu, select Report.

Enter your data in the white text field **1**. If the “Add report” option is active in the system settings (on page Examination, Settings, Resting ECG, Diagnostic, Report), an automatic report is already present in the text field. When you click on Confirm **2** your input is saved and the evaluation gets the “Confirmed” status (see “End dialogue”).

In the event that your report text is not yet complete but you want to save it nevertheless, without reaching “Confirmed” status, reset the evaluation “Confirmed” status in the “End dialogue”.

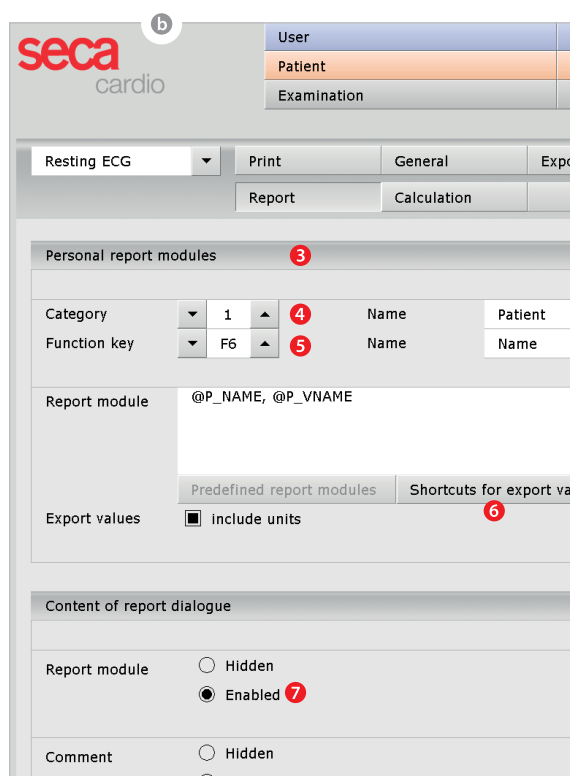


b Text modules for writing reports

On the page Examination, Resting ECG, Settings, Diagnostic, Report text modules for writing a report can be created **3**. A total of four groups **4** with up to eight text modules **5** can be stored. The text modules are called in the “Report dialogue” using keyboard commands (F5 to F12).

A report module can be created from normal text as well as variables. When using a report module in the “Report dialogue”, the actual value from the evaluation is used instead of the variable and is automatically inserted in the report text. The structure of a variable is @VARIABLE (e.g. Heart Rate, Resting: @HF_R). The Shortcuts for export values **6** button provides you with a list containing all the available variables. If the report modules should be shown in the “Report dialogue”, make sure that the Enable option **7** is activated. Alternatively, the text modules can be shown in the “Report dialogue” via the Show modules button.

There is also the option of entering a text or a user-defined report text (also as normal text and variables), which will be automatically shown in each report **8**. The predefined text can be changed later in the “Report dialogue”.

Save your input.

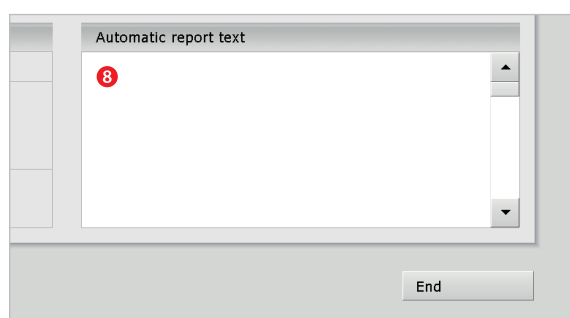

06.4 Ending the evaluation

The evaluation is closed with the **End** button. The end dialogue is opened. The status of an examination can be changed here ❶.

Confirmed ❷

A confirmed evaluation can be reset to “Not confirmed” by deselecting the “Confirmed” option (e.g. if reporting has not been completed).


Printed ❸

Indicates if the evaluation has been printed.

Locked ❹

After reporting has been completed by an authorised person, set the status of the evaluation to “Locked”. After that, the evaluation can still be viewed, but no longer changed.

Click on **Confirm** ❺ to close the examination.

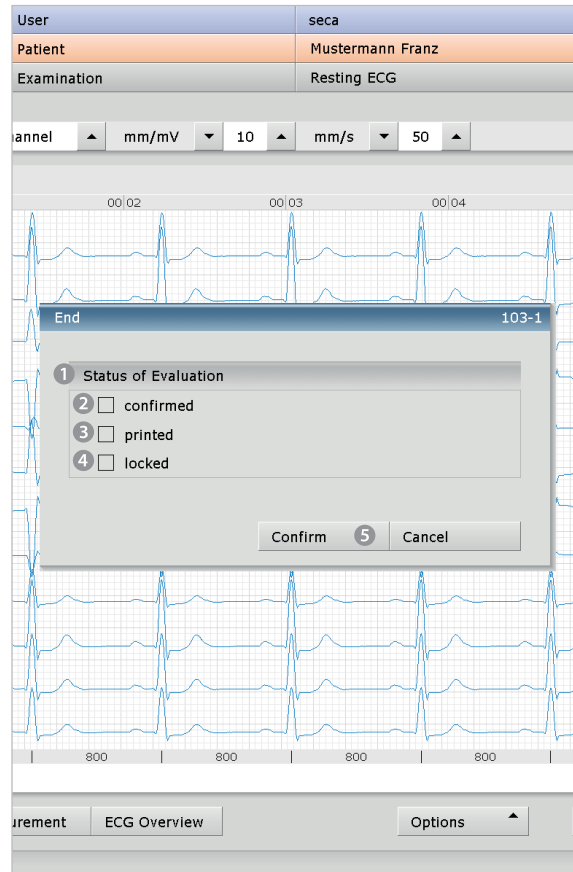

07.1 Technical data and system requirements

Technical data of seca CT 320/321		
Frequency response	0.05 – 500 Hz	
Accuracy	< 1.5%	
Time constant	3.2 sec	
Sampling rate	1.0ms / 1000 Hz for extremities and chest wall 0.25ms / 4000 Hz for pacemaker	
A/D converter	24 bit	
Input impedance	> 50 MΩ	
Amplitude quantification	0.3 µV/bit	
CMRR	> 70 dB	
Impedance measurement *)	At all electrode leads (not N) with automatic quality indication	
Defibrillation protection	Electrical strength 5000 V Recovery time after defibrillation < 10s	
Voltage supply	USB devices	USB cable
	BT devices	Rechargeable lithium-ion battery
Energy consumption	Max. 2.5 watt during recording	
IT connection	USB (cable length 2.50 m) or Bluetooth (range up to 80 m, depending on ambient conditions)	
Software, additional functions (seca cardio 150)	Power filter, muscle filter, AD filter (Anti-drift), pacemaker recognition	
Operating conditions	Temperature	+10°C ... +40°C
	Air humidity	25 ... 95% rH
	Air pressure	700 ... 1060 hPa
Transport and storage conditions	Temperature	-20°C ... +45°C
	Air humidity	25 ... 95% rH
	Air pressure	700 ... 1060 hPa
Size	approx. 160 * 85 * 25 mm (L * W * H)	
Weight	approx. 300 g	
Patient leads	approx. 1050 mm (extremities)	
	approx. 700 mm (chest)	
Classification	Safety class II	(seca CT 320)
	Safety class III	(seca CT 321)
	Class IIa	
	Type BF defibrillation protected (not intracardiac)	
	IEC 60601-1	
	IEC 60601-2-25	
	IEC 60601-2-51	

*) Does not apply to Bluetooth devices



System requirements	
Operating system	<p>Windows XP (x64)</p> <p>Windows Vista (x64)</p> <p>Windows 7 (x64)</p> <p>Windows Server 2003 (x64)</p> <p>Windows Server 2008 (x64)</p> <p>Windows Server 2008 R2</p> <p>Older versions are not supported.</p>
PC	<p>The PC hardware must meet the minimum requirements of the operating system used.</p> <p>seca cardio 150 requires additional RAM (256 MB).</p> <p>Ensure that there is enough free hard disk space for the seca cardio 150 evaluations.</p> <p>File size of the evaluations:</p> <p>Resting ECG approx. 200 KB (approx. 10 sec ECG)</p>
Hardware & ports	<p>DVD or CD-ROM drive</p> <p>USB port</p>

07.2 Manufacturer declaration on EMC (Electromagnetic Compatibility) according to IEC 60601-1-2:2007

Cable lengths of the patient leads and the USB cables

Patient leads seca CT 320/321 approx. 1050 mm & 700 mm

USB cable seca CT 320/321 approx. 2500 mm

Manufacturer's Declaration – electromagnetic emissions

The seca CT 320/321 ECG device is designed for operation in the electromagnetic environment stated below. The customer or user of seca CT 320/321 should make sure that it is used in such an environment.

Emission Measurements	Compliance	Electromagnetic Environment - Guidelines
HF emissions according to CISPR11	Group 1	seca CT 320/321 uses HF energy only for its internal function. Its level of HF emission is therefore very low and is unlikely to be sufficient to interfere with other electronic devices.
HF emissions according to CISPR11	Class B	seca CT 320/321 is suitable for use in all establishments, including domestic establishments and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes.
Harmonics according to IEC61000-3-2	Not applicable	
Voltage fluctuations/flickers according to IEC61000-3-3	Not applicable	

Manufacturer's Declaration – electromagnetic immunity


The seca CT 320/321 ECG device is designed for operation in the electromagnetic environment stated below. The customer or user of seca CT 320/321 should make sure that it is used in such an environment.

Immunity Tests	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment - Guidelines
Electrostatic discharge (ESD) according to IEC 61000-4-2	± 6 kV contact discharge ± 8 kV air discharge	± 6 kV contact discharge ± 8 kV air discharge	Floors should be made of wood or concrete or be equipped with ceramic tiles. If the floor is provided with synthetic material, the relative air humidity must be at least 30 %.
Fast transient electric interference factors/bursts according to IEC 61000-4-4	± 2 kV for net wires ± 1 kV for input and output wires	Not applicable	The quality of the supply voltage should correspond to the one of a typical business or clinical environment.
Surges according to IEC 61000-4-5	± 1 kV push-pull voltage ± 2 kV push-push voltage	Not applicable	The quality of the supply voltage should correspond to the one of a typical business or clinical environment.
Voltage drops, short-time interruptions and fluctuations in the supply voltage according to IEC 61000-4-11	< 5 % U_T for 0.5 period (> 95 % drop) 40 % U_T for 5 periods (60 % drop) 70 % U_T for 25 periods (30 % drop) < 5 % U_T for 5 s (> 95 % drop))	Not applicable	The quality of the supply voltage should correspond to the one of a typical business or clinical environment. If the user of seca CT 320/321 requires continued function, even if interruptions in the energy supply occur, it is recommended to supply seca CT 320/321 from an interruption-free power supply.
Magnetic field with supply frequency (50/60 Hz) according to IEC 61000-4-8	3 A/m	3 A/m	Magnetic fields with net frequency should correspond to the typical values, as they can be found in the business and clinical environment.

COMMENT: U_T is the net AC voltage before applying the test levels

Manufacturer's Declaration – electromagnetic emissions

The seca CT 320/321 ECG device is designed for operation in the electromagnetic environment stated below. The customer or user of seca CT 320/321 should make sure that it is used in such an environment.

Immunity Tests	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment - Guidelines
Conducted HF transients according to IEC 61000-4-6	3 V _{effective} value 150 KHz to 80 Mhz	[U ₁] V 10 V _{effective} value	<p>Portable and mobile radio sets should not be used at a closer distance to the device including the leads than the recommended protective distance which is determined according to the equation of transmitting frequency.</p> <p>Recommended protective distance:</p> $d = (3.5/U_1) \sqrt{P}$ $d = (3.5/E_1) \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = (7/E_1) \sqrt{P} \quad 800 \text{ MHz to } 2.5 \text{ GHz}$ <p>with P as the nominal power of the transmitter in watt (W) according to the indications of the transmitter manufacturer and d as the recommended protective distance in meters (m). According to an examination on-site a) the field strength of stationary radio transmitters should be inferior to the compliance level b) with regard to all frequencies. In the vicinity of devices carrying the following symbol, interferences are possible:</p> 
Radiated HF transients according to IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	10 V/m	

COMMENT 1:

With 80 Hz and 800 MHz the higher frequency range is valid.

COMMENT 2:

These guidelines may not apply in every case. The propagation of electromagnetic variables is influenced by absorptions and reflections of buildings, objects and people.

a) The field strength of stationary transmitters, such as e.g. base stations of mobile phones and mobile transmitting stations, amateur radio stations, AM and FM broadcasting as well as television networks cannot be exactly predetermined theoretically. In order to determine the electromagnetic environment regarding the stationary transmitters, a study of the location should be considered. If the measured field strength exceeds the above-mentioned compliance levels at the location where the device is used, the device should be watched in order to prove the intended functions. If unusual performance features are observed, it may be necessary to take additional measures, for example reorienting or relocating the device.

b) Over the frequency range from 150 kHz to 80 MHz the field strength should be inferior to 10 V/m.

Recommended protective distances between portable and mobile HF telecommunication devices and seca CT 320/321

seca CT 320/321 is designed for the operation in an electromagnetic environment in which the HF transients can be controlled. The user can help avoid electromagnetic interferences by maintaining the minimum distance between portable and mobile HF telecommunication devices (transmitters) and the device – depending on the initial performance of the communication device – as indicated below.

Nominal power of the transmitter W	Protective distance depending on the transmitting frequency in m		
	150 kHz to 80 MHz $d = (3.5/U1) \sqrt{P}$	80 MHz to 800 MHz $d = (3.5/E1) \sqrt{P}$	800 MHz to 2.5 GHz $d = (7/E1) \sqrt{P}$
0.01	0.04	0.04	0.08
0.1	0.11	0.11	0.22
1	0.35	0.35	0.70
10	1.11	1.11	2.22
100	3.50	3.50	7.00

For transmitters whose maximum nominal power is not indicated in the above table, the recommended protective distance d can be determined in meters (m), using the equation affiliated with the corresponding column. P is the maximum nominal power of the transmitter in watt (W) according to the indications of the manufacturer of the transmitter.

COMMENT 1:

With 80 MHz and 800 MHz the higher frequency range is valid.

COMMENT 2:

These guidelines may not apply in every case. The propagation of electromagnetic variables is influenced by absorptions and reflections of buildings, objects and people.

07.3 EC Declaration of Conformity



Konformitätserklärung
declaration of conformity
Certificat de conformité
Dichiarazione di conformità
Declaración de conformidad
Överensstemmelsesattest
Försäkran om överensstämmelse
Konformitetserklæring
vaatimuksenmukaisuusvakuutus
Verklaring van overeenkomst
Declaração de conformidade
Δήλωση Συμβατότητας
Prohlášení o shodě
Vastavusdeklaratsioon
Megfelelőségi nyilatkozat
Atitikties patvirtinimas
Atbilstības apliecinājums
Oświadczenie o zgodności
Izjava o skladnosti
Vyhlásenie o zhode
Onay belgesi

Das 12-Kanal-PC-EKG-Gerät
The 12-channel PC ECG
L'appareil ECG 12 canaux connectable à un ordinateur
L'elettrocardiografo 12 canali per PC
El aparato PC ECG de 12 canales
12-kanal-pc-EKG-apparatet
12-kanals-PC-EKG-apparat
PC-EKG-apparatet med 12 kanaler
12-kanavainen PC-EKG-laite
Het 12-kanaals PC-ECG-apparaat
O aparelho de ECG para PC de 12 canais
Η συσκευή ΗΚΓ 12 καναλιών Η/Υ
12-kanálový PC EKG přístroj
12 kanāliņa EKG-arvutiseade
A 12 csatornás számítógépes EKG készülék
12 kanālu EKG datoram pieslēdzama ierīce
12 kanālu datora EKG ierīce
12-kanalowy komputerowy aparat EKG
12-kanalna PC-EKG naprava
12-kanálový PC EKG přístroj
12 Kanal PC-EKG Cihazı

seca 320



Konformitätserklärung
 declaration of conformity
 Certificat de conformité
 Dichiarazione di conformità
 Declaración de conformidad
 Overensstemmelsesattest
 Försäkran om överensstämmelse
 Konformitetserklæring
 vaatimustenmukaisuusvakuutus
 Verklaring van overeenstemming
 Declaração de conformidade
 Δήλωση Συμβατότητας
 Prohlášení o shodě
 Vastavusdeklaratsioon
 Megfelelőségi nyilatkozat
 Atitikties patvirtinimas
 Atbilstības apliecinājums
 Oświadczenie o zgodności
 Izjava o skladnosti
 Vyhlásenie o zhode
 Onay belgesi

Das 12-Kanal-PC-EKG-Gerät
 The 12-channel PC ECG
 L'appareil ECG 12 canaux connectable à un ordinateur
 L'elettrocardiografo 12 canali per PC
 El aparato PC ECG de 12 canales
 12-kanal-pc-EKG-apparatet
 12-kanals-PC-EKG-apparat
 PC-EKG-apparatet med 12 kanaler
 12-kanavainen PC-EKG-laite
 Het 12-kanals PC-ECG-apparaat
 O aparelho de ECG para PC de 12 canais
 Η συσκευή ΗΚΓ 12 καναλιών Η/Υ
 12-kanalový PC EKG přístroj
 12 kanaliga EKG-arvutiseade
 A 12 csatornás számítógépes EKG készülék
 12 kanālu EKG datoram pieslēdzama ierīce
 12 kanālu datora EKG ierīce
 12-kanalowy komputerowy aparat EKG
 12-kanalna PC-EKG naprava
 12-kanalový PC EKG přístroj
 12 Kanal PC-EKG Cihazı

seca 321

07.4 Shutdown, storage and transport

Putting out of operation and storage

Clean and disinfect seca CT 320/321 and its components before putting it out of operation.



Make sure that the storage location is dust-free, dry and away from direct sunlight.

After a long period of non-operation, seca CT 320/321 may only be used again if a technical safety check has been carried out by your authorised seca dealer.

Transport

Clean and disinfect seca CT 320/321 and its components before transport.



Use the original packaging for transport. This is a sensitive piece of electronic equipment.

If the original packaging is not available, pack the device in a way that it is protected against collision, humidity and dust.



The device must comply with the operating conditions when it is put into operation again, e.g. operating temperature (*see 07.1 Technical Data...*).


Ambient conditions for storage and transport

Temperature: -20° ... +45°C
Air humidity: 25 ... 95% rH
Air pressure: 700 ... 1060 hPa



07.5 Disposal

The device and all its components must be disposed of in a proper manner in compliance with applicable regulations (that is, in accordance with the valid laws governing electrical and electronic waste). The device must not be disposed of as normal domestic waste.

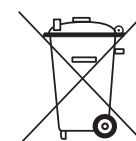

The original packaging is recyclable (cardboard/waste paper).



07.6 Keyboard navigation and shortcuts

Use the quick links in the main navigation, the keyboard navigation and the keyboard shortcuts to enable fast and convenient working.

Quick links in the main navigation

seca cardio	User	seca	1		?	_	×
	Patient		2	2			
	Examination		3	3			▼

LEFT-CLICK

- 1 User master data
- 2 Call last patient
- 3 Examination main menu

RIGHT-CLICK

- 1 Evaluation search
- 2 Call last patient
- 3 Evaluation last displayed

seca cardio	User	seca	4		?	_	×
	Patient	Mustermann Franz	5	4	10.10.1960 (51 Y.)		
	Examination	Resting ECG	6	5			▼

LEFT-CLICK

- 5 User master data
- 4 Patient master data
- 6 Examination main menu

RIGHT-CLICK

- 4 All the patient's evaluations
- 5 Evaluation last displayed of this examination

Keyboard navigation

When you press the Alt key, the initial letter of all the buttons on a screen page is underlined. Pressing an initial letter in combination with the Alt key triggers the corresponding button.

	User	seca	?	_	×
	Patient				
	Examination	Resting ECG			▼
New Resting ECG					
Show Evaluation					
Show Comparison					
Assign Evaluation					
Settings					

Generally accepted keyboard shortcuts



End, Cancel



Confirm, continue



Program Information



User main menu



Patient main menu



Examination main menu



Patient master data of the selected patient



All examinations of the selected patient



List of recently opened evaluations
(same as clicking on the arrow button
at top right)



List of recently opened evaluations










Evaluation search








Create system report, service e-mail

Generally accepted keyboard shortcuts for open evaluation

	Report dialogue
	Medication input dialogue
	Call comparison ¹⁾
	Call trend ¹⁾
	Call print ¹⁾
	Call settings ¹⁾
	Call Options menu ¹⁾

Resting ECG keyboard shortcuts during the recording

	Start the recording
	End the recording
	Increase amplitude
	Decrease amplitude
	Blood pressure input dialogue ²⁾

¹⁾ Keyboard shortcuts will only work if the corresponding button is available on the screen page.
²⁾ This keyboard shortcut is also available in the evaluation display.

Medical Scales and Measuring Systems since 1840

seca gmbh & co. kg
Hammer Steindamm 9–25
22089 Hamburg · Germany
Telephone +49 40 20 00 00 0
Fax +49 40 20 00 00 50
info@seca.com

All contact data under www.seca.com